ENGLISH

INTENDED USE/INDICATIONS:

The intended use of the Pinnacle Cervical Collars is to provide stabilization and motion restriction for the treatment of trauma-related injury, acute post-surgical recovery, and the management of chronic conditions of the cervical spine. Indicated for post-surgical stabilization, cervical disc syndrome, cervical stenosis, fracture management, sprains / strains, decompressive procedures, cervicogenic headaches, radiculopathy, trauma / whiplash.

INTENDED USERS:

The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the collar, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in instructions.
- For single patient use only.
- Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
- Follow the licensing healthcare practitioner's protocol regarding method, frequency and duration of administration.

INSTRUCTIONS FOR USE:

BACK PANEL PLACEMENT

1. Make sure the patient's head is in a neutral position.
2. Slide the back panel under the neck.
3. Align the loop straps between the ears and shoulders, centered along the neck's midline.

FRONT COLLAR PLACEMENT

4. Band the back ends of the front collar inward to fit the neck.
5. Start with the chin piece at its lowest position.
6. Place the bottom of the front collar against the chest and align it with the back panel straps.

CHIN ADJUSTMENT

7. Pull out and turn the knob clockwise to raise the chin piece until the chin is supported and the head is neutral.
8. Lock the knob into a stop position.

STRAP ATTACHMENT

9. Attach the straps to the hooks on the front collar sides.
10. Make sure both straps are evenly tightened for a snug fit.

Tips:

The chin piece should not press on the throat. Adjust side panels if needed. Trim excess strap material if necessary.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Cotton, DMNT Resin, High Density Polyethylene (HDPE), Nylon

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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