PINNACLE CERVICAL COLLAR TX 174

MODEL #SP40174-000

INSTRUCTIONS FOR USE

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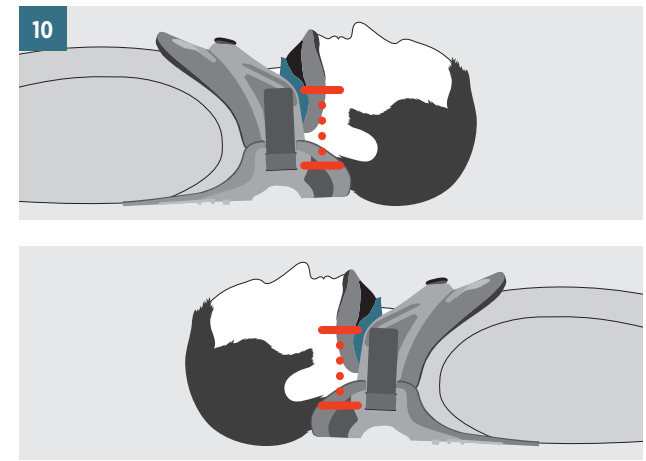
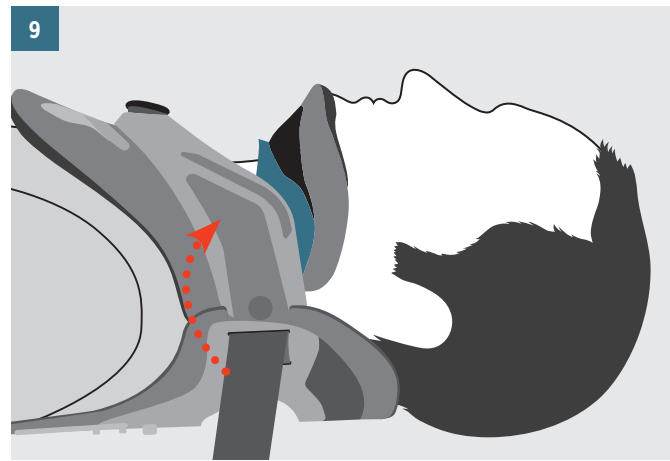
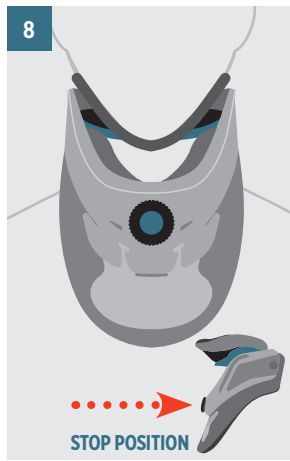
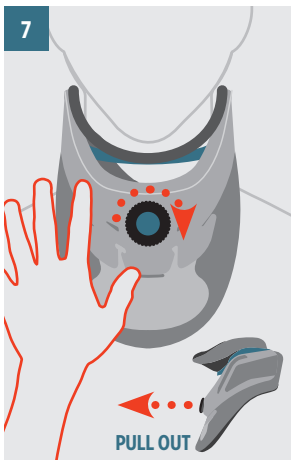
ENGLISH

CHIN ADJUSTMENT

7. While holding the front collar in position, **pull out and rotate** the knob clockwise to raise the chin piece until the patient's chin is supported and head is in a neutral position.
8. Once adjustments have been made, ensure the knob is secured into one of the stop positions.

STRAP ATTACHMENT

9. Attach straps to the hook on the sides of the front collar.
10. Ensure both straps have an equal amount of tension and proper alignment is maintained, resulting in a snug and supported fit.

NOTE: Ensure that chin piece is not pressing on patient's throat. If it is, adjust side panels of the front collar so they are off the shoulders.

NOTE: Excess strap material can be trimmed to desired length.

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 WARNINGS 

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the collar, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: This device is offered for sale by or on the order of a licensed health care professional.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

CARE & WASHING INSTRUCTIONS

Hand wash pads in cold water with a mild detergent, rinse and air dry.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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©2023 Breg, Inc. All rights reserved.