1

Loosen laces, and slide foot into the brace. Position heel in the hole at the rear and bottom of the brace. Lace the ankle brace, and tie snugly.

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2

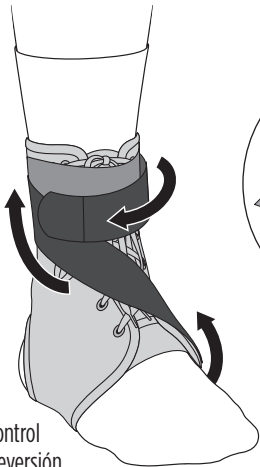
When the brace feels comfortable, wrap the elastic top strap over laces, and secure. Re-adjust if necessary.

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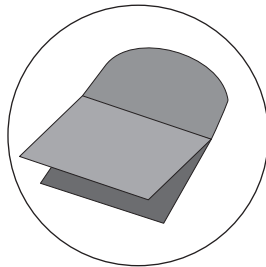
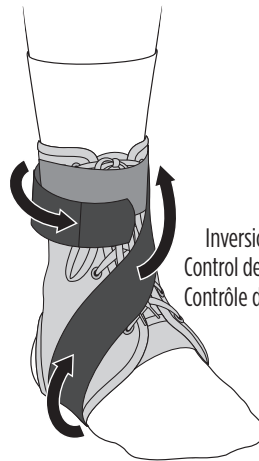
Right foot shown:

For additional **eversion control**, wrap strap from the bottom of the brace around the **medial side** of the foot, and secure around the top of the ankle. For additional **inversion control**, wrap strap from the bottom of the brace around the **lateral side** of the foot, and secure around the top of the ankle. Adjust the **Reversal Y-Tab** at the end of the long strap if needed.

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VersaStrap Ankle Brace Instructions for Use

Example illustrates right foot brace application

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WARNINGS

WARNING: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

WARNING: If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

WARNING: This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

CAUTION: Follow application and care process described in instructions.

CAUTION: For single patient use only.

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Breg, Inc.