

Curtis Shoulder Cuff-Full ROM

Instructions for Use:

Application Procedure


1. Measure circumference of chest and select proper size on chart below. Place affected arm through arm hole and secure with tab. Wrap vest around torso and secure with two pressure sensitive closures.
2. Attach lower torso strap to lower buckle and tighten securely.
3. Secure lower arm control strap to upper buckle and adjust for arm lift control.
4. Attach upper arm control strap and secure with buckle below unaffected arm and adjust.
5. Tuck remaining straps inside their respective neoprene sleeves, to cover buckle. Trim excess strap material with scissors.

Use and Care:

- Hand wash with a mild detergent in lukewarm water.
- Be sure to rinse well since soap residue may cause skin irritation.
- Lay flat, Air dry.

Ordering Information

| Size | Measurements |
|------|--------------|
| S | 34" - 38" |
| M | 38" - 42" |
| L | 42" - 46" |
| XL | 46" - 50" |

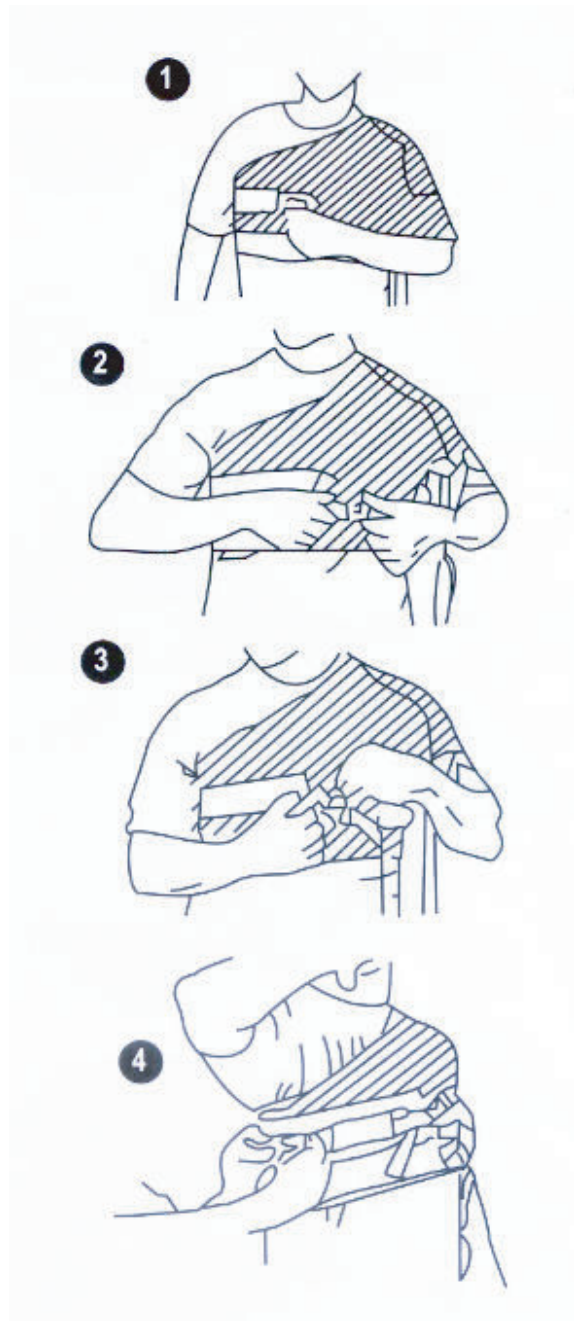

WARNING: This device is intended for use only under the direction of a medical professional. If you experience any pain, swelling, sensation changes or any unusual reactions while using this product, consult your medical professional immediately.



Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com