



AFO Attachment for Thruster RLF

INSTRUCTIONS FOR USE AW-CP020155 REV. J 0723

ANKLE FOOT ORTHOSIS



This accessory is offered for sale by or on the order of a physician or other qualified medical professional. This accessory is not intended for reuse on a second patient. This accessory is generally not intended for patients in excess of 250 lbs (114 kilos).

Indications: Indicated to maintain hinge position for proper suspension of the Thruster RLF Medial and Thruster RLF Lateral when inadequate calf musculature prevents suspension of the brace alone.

WARNING: Do not leave this accessory in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this accessory, discontinue use and seek medical attention.**

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.



© 2023 Breg, Inc. All Rights Reserved
2382 Faraday Avenue, Suite 300
Carlsbad, CA 92008 USA
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440
Fax: +1-760-795-5295
www.Breg.com



EC REP


E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany



AFO Attachment (Ankle Foot Orthosis) INSTRUCTIONS FOR USE




1. Pull back the pad on the inside of the lower shell of the Thruster RLF Brace (LA or MA). Apply the AFO Attachment to the outside of the upright, and insert the screws from the kit through the holes. Attach the screws to the threaded holes from the inside of the upright, as indicated in photo, into the extension upright of the AFO attachment.

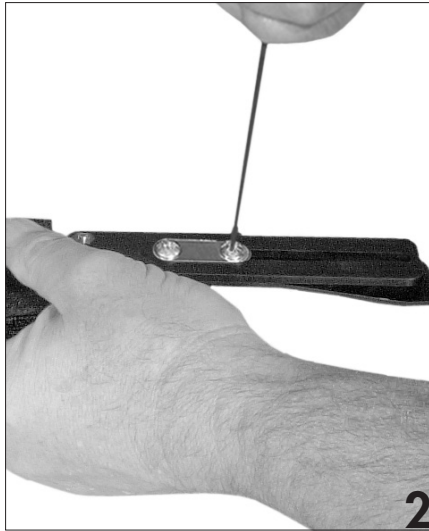


Loosen the two adjustment screws to permit sliding of the adjustment slot. **Note:** The 3/32 ball driver provided with the brace, or the 3/32 hex driver provided with the AFO Kit can be used for adjustment.



Apply and adjust the brace to the patient according to the brace application instructions. After fitting the brace, firmly tighten the adjustment screws.

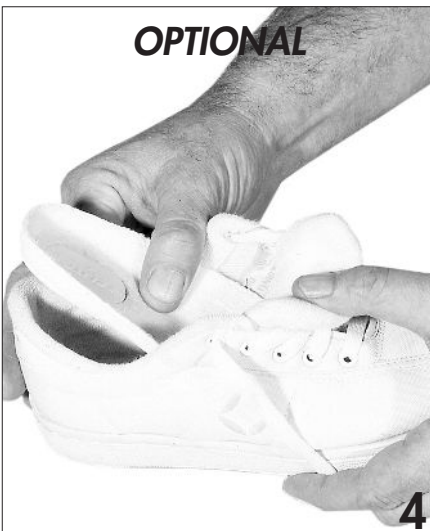

Note: If desired, the AFO Attachment may be attached to the shoe with hook and loop fasteners. To install the shoe fastener, pull back the padded shoe insole.

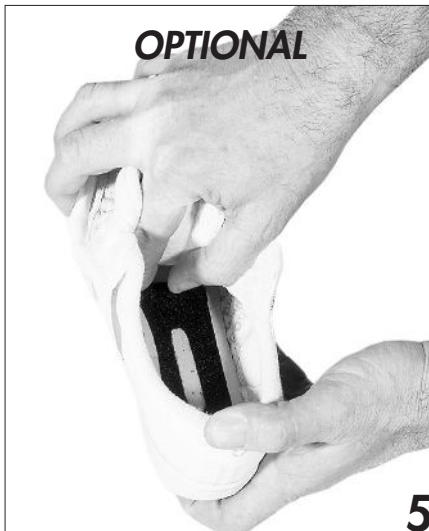

Peel the protective cover from the adhesive on the loop insert and press the adhesive while centering to the heel area of the shoe.



(Shown: Aligner Brace) The hook and loop fastener can be engaged by pressing the hook on the bottom of the footplate to the loop insert in the shoe. **Note: The padded shoe insole must be placed over the footplate.**