
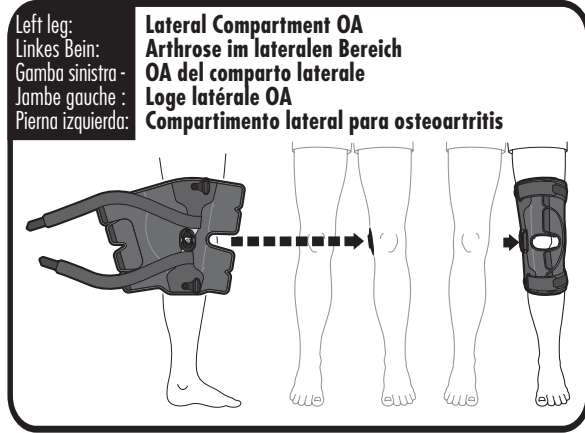
Right leg:
Rechtes Bein:
Gamba destra -
Jambe droite :
Pierna derecha:

Lateral Compartment OA
Arthrose im lateralen Bereich
OA del comparto laterale
Loge latérale OA
Compartimento lateral para osteoartritis

Left leg:
Linkes Bein:
Gamba sinistra -
Jambe gauche :
Pierna izquierda:

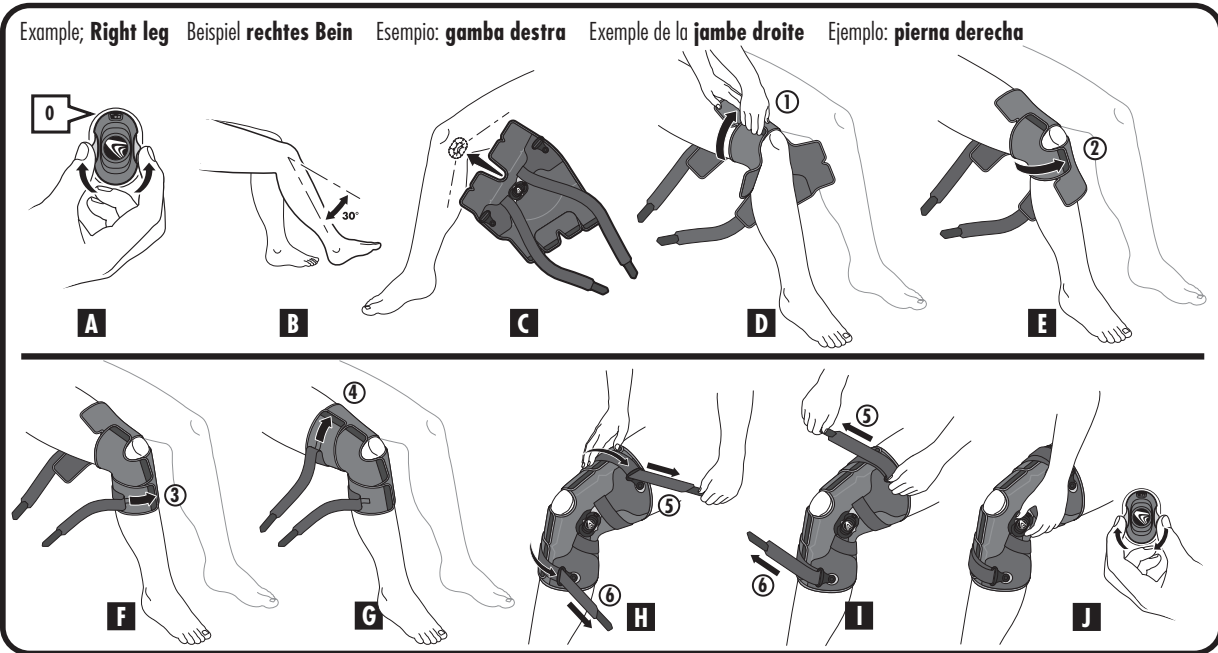Lateral Compartment OA
Arthrose im lateralen Bereich
OA del comparto laterale
Loge latérale OA
Compartimento lateral para osteoartritis

English

- A)** Prior to application, set the hinge to 0.
B) Sit with the knee bent at 30°
C) Position the brace by placing the hinge on the inside of the knee.
D) Maintain the brace position with one hand while using the other hand to pull tab #1 across the thigh above the knee and fasten securely.
E) Pull tab #2 across the shin below the knee and fasten securely.
F) Pull tab #3 across the lower shin and fasten securely.

- G)** Pull tab #4 across the upper thigh and fasten securely.
H) Pass strap #5 through the plastic ring on the inside of the thigh.
I) To avoid rotation of the brace during application, secure the brace with one hand, as shown, while tensioning strap #5. Lay the strap onto itself and fasten securely. Repeat for strap #6.
J) Once the brace is properly applied, pain relief may be achieved by turning the thumbwheel hinge dial clockwise to increase pressure to the inside of the knee.

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WARNING: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS. FAILURE TO PROPERLY POSITION THE BRACE AND FASTEN EACH STRAP WILL COMPROMISE PERFORMANCE AND COMFORT.

WARNING: FOR SINGLE PATIENT USE ONLY.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE

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TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: CARE AND CLEANING OF THIS PRODUCT IS ESSENTIAL TO ITS CONTINUING STRENGTH AND PERFORMANCE. FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED IN INSTRUCTIONS.

CAUTION: WHEN USING BRACES BILATERALLY, MODIFY ACTIVITY AS NECESSARY TO KEEP MEDIAL HINGES FROM CONTACTING EACH OTHER.

CAUTION: WHEN USING THE FREESTYLE OA BRACE IT IS IMPORTANT TO KEEP DIRT AND GRIT OUT OF THE HINGE MECHANISM TO MAINTAIN PROPER HINGE MOBILITY AND BRACE FUNCTION.

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Freestyle™ OA Brace for Lateral OA Application

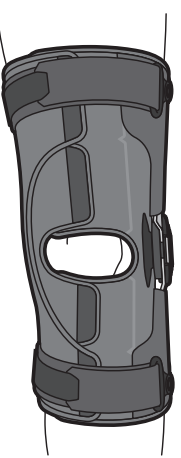
INSTRUCTIONS FOR USE

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BREG

2382 Faraday Avenue, Suite 300
Carlsbad, CA 92008 U.S.A.
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440
Fax: +1-760-795-5295
www.breg.com

© 2023 Breg, Inc. All rights reserved.

AW-1.00289 Rev F 07/23

CE
EC REP

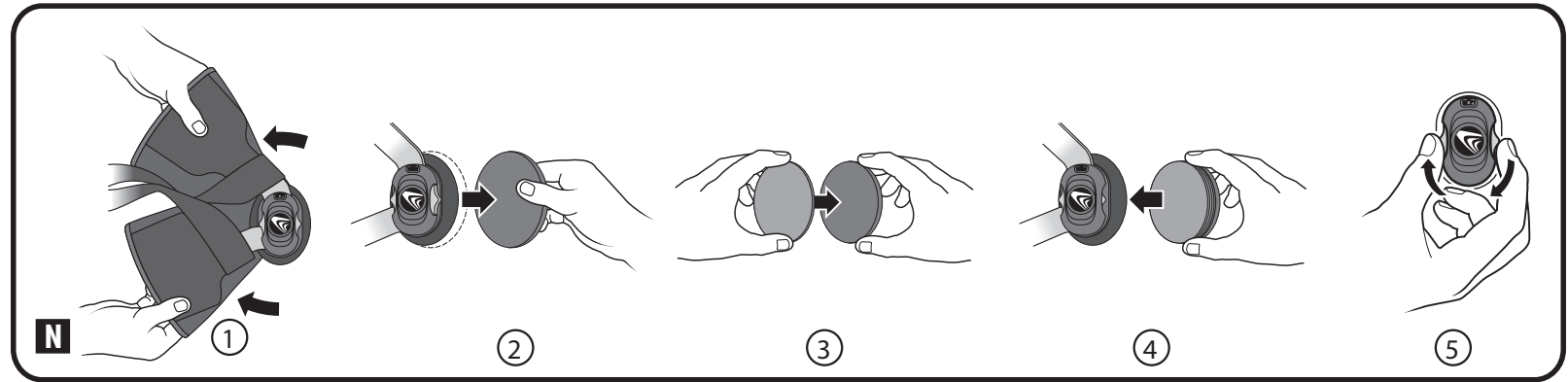
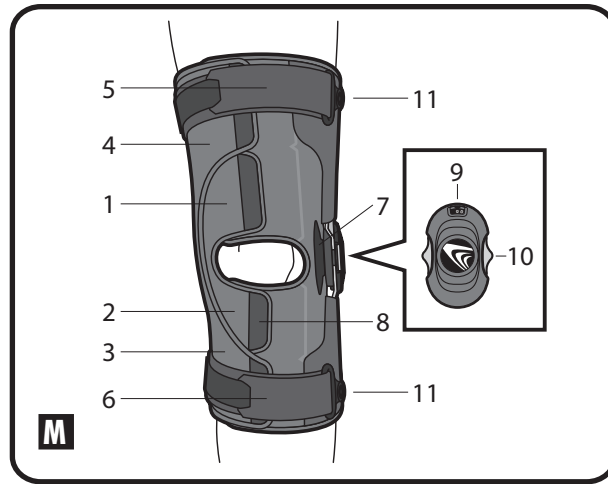
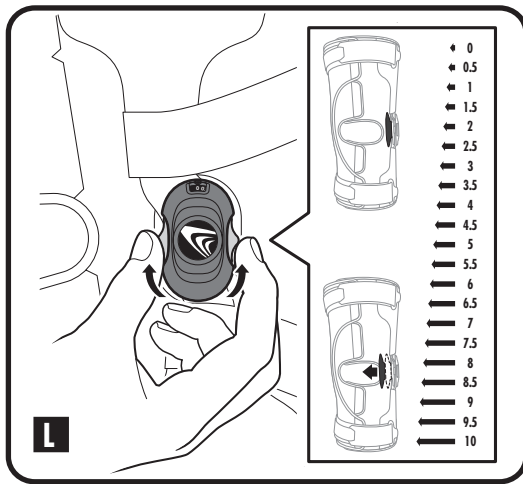
E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

MD

Scan QR Code for Instruction Video

breg.com/foa

English

A) INSTRUCTION VIDEO

To view a video demonstrating how to use the brace, scan the QR code or go to the website:

www.breg.com/foa

B) HINGE ADJUSTMENT

With the brace properly applied, OA pain symptoms can be relieved by adjusting the thumbwheel hinge dial. Turn the thumbwheel hinge dial clockwise to increase pressure and counterclockwise to reduce the pressure on the inside of the knee. The hinge settings window displays settings ranging from 0 to 10mm. The display setting will increase or decrease as you adjust the thumbwheel hinge dial.

Make sure a large gel condyle pad is attached to the hinge.

Always follow your medical professional's recommendation regarding the proper hinge setting.

C) PRODUCT DIAGRAM

- 1: Tab #1
- 2: Tab #2
- 3: Tab #3
- 4: Tab #4
- 5: Strap #5
- 6: Strap #6
- 7: Condyle pad
- 8: Finger Pockets
- 9: Hinge Settings Window
- 10: Thumbwheel Hinge Dial
- 11: Plastic Rings

NOTICE TO USERS

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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