
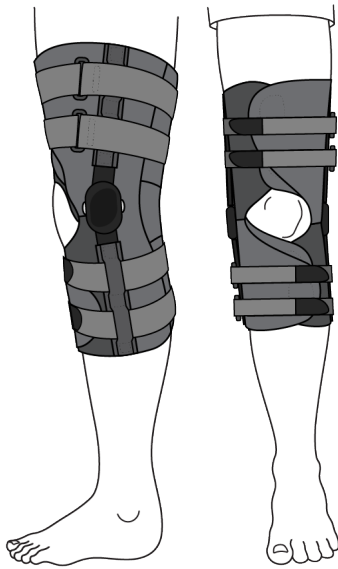
# CrossRunner™ & CrossRunner Short

Wraparound  
Instructions For Use

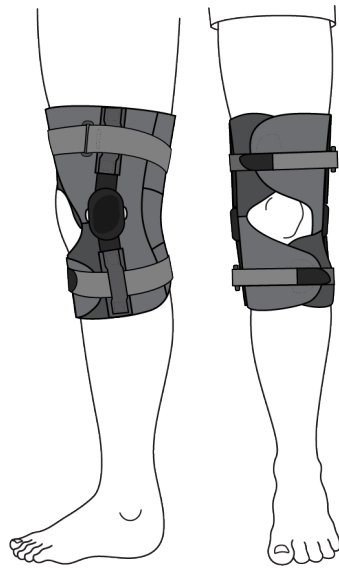CrossRunner


CrossRunner Short

2382 Faraday Avenue, Suite 300  
Carlsbad, CA 92008 USA  
Toll Free Tel: 1-800-321-0607  
Local: +1-760-795-5440  
Fax: +1-760-795-5295  
[www.breg.com](http://www.breg.com)

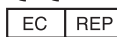©2023 Breg, Inc. All rights reserved.  
CrossRunner is a trademark of Breg, Inc.

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

# CrossRunner & CrossRunner Short Knee Brace

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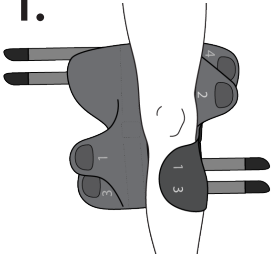
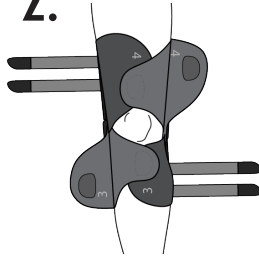
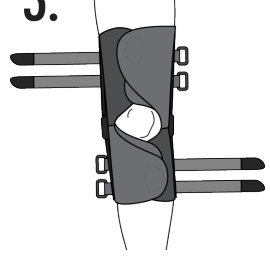
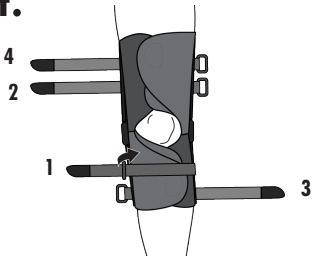
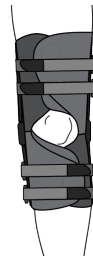
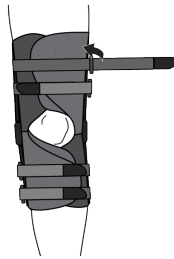
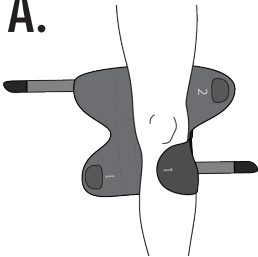
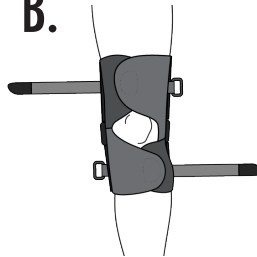
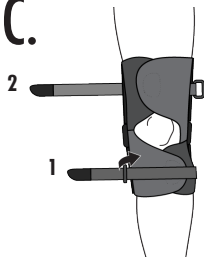
## Warnings

**⚠ WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

**⚠ WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

**CAUTION:** UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

**CAUTION:** FOR SINGLE PATIENT USE ONLY.

**CAUTION:** CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS. FAILURE TO PROPERLY POSITION THE BRACE AND FASTEN EACH STRAP WILL COMPROMISE PERFORMANCE AND COMFORT.

**CAUTION:** USE IN HIGH-IMPACT ACTIVITIES MAY CAUSE PREMATURE WEAR OF COMPONENTS.

**CAUTION:** CARE AND CLEANING OF THIS PRODUCT IS ESSENTIAL TO ITS CONTINUING STRENGTH AND PERFORMANCE. FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED IN INSTRUCTIONS.

## Care and Cleaning

- ATTACH STRAP ENDS TO STRAP PRIOR TO CLEANING.
- AFTER EXPOSURE TO SALT WATER OR DIRT, THOROUGHLY RINSE THE BRACE WITH FRESH WATER AND ALLOW TO AIR DRY.
- A DRY LUBRICANT, SUCH AS TEFLON SPRAY, MAY BE USED ON THE HINGE OCCASIONALLY. DO NOT USE WET LUBRICANTS.
- THE HINGE ASSEMBLIES MAY BE REMOVED TO ALLOW EASIER WASHING OF THE SLEEVE AND STRAPS.
- THE SIDE BARS MAY BE BENT TO ACCOMMODATE ANATOMY. BEND IN EQUAL AMOUNTS ABOVE AND BELOW THE HINGE.
- PERIODICALLY INSPECT THE SLEEVE, STRAPS AND HINGES FOR EXCESSIVE WEAR. BE SURE STOPS ARE IN PLACE.

## Range of Motion (ROM) Stop

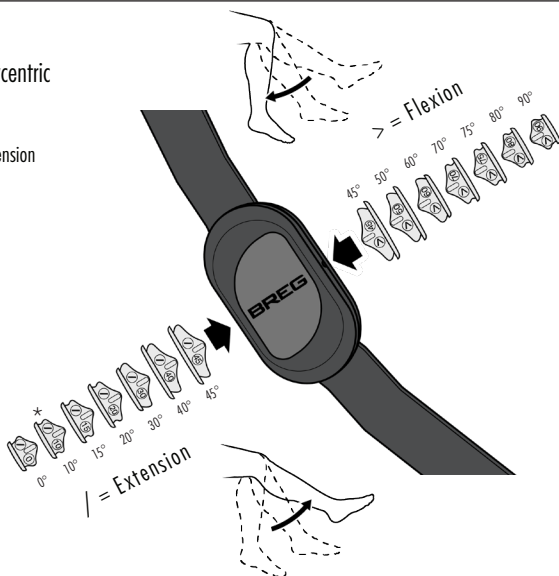
## CrossRunner Adjustment Instructions — Polycentric hinge with Stops:

Dual function extension and flexion ROM Stops

- Every Stop may be used to control Flexion or Extension
- 7 Unique Stops provide 14 angle set positions:
  - 80 Flexion / 10 Extension (Pre-Installed)\*
  - 45 Flexion / 45 Extension
  - 50 Flexion / 40 Extension
  - 60 Flexion / 30 Extension (Quantity 2)
  - 70 Flexion / 20 Extension
  - 75 Flexion / 15 Extension
  - 90 Flexion / 0 Extension (Quantity 2)



ROM Stop Removal Key


## ROM Stop Removal

**⚠ Warning:** Do not point ROM Stops at anyone's head or face during removal. The Removal Key or a small screwdriver can remove the Stops from the hinge.

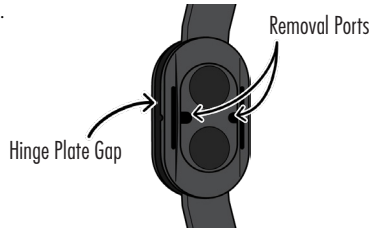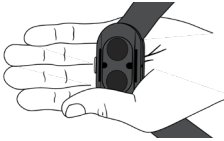
- Grip the hinge as shown then follow either removal method below

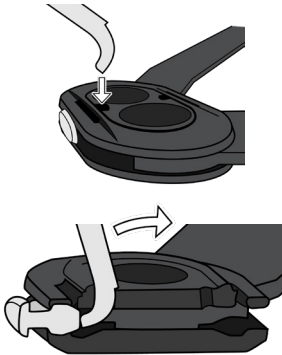- Removal port method
  - Insert the ROM Stop Removal Key into the port
  - Leverage the Key to partially eject ROM Stop
  - Remove the Stop manually

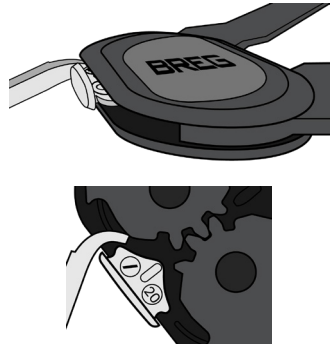- Hinge plate gap method
  - Insert the ROM Stop Removal Key into the hinge plate gap by the ROM Stop
  - Rotate the Key to eject the ROM Stop


## Insertion of the ROM Stops

- Select the ROM Stop
  - Extension — Anterior side
  - Flexion — Posterior side
- Press firmly to snap in ROM Stop

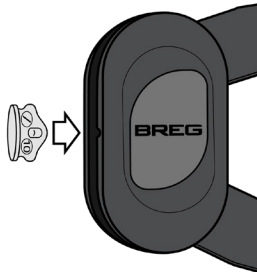Extension

Flexion

IF ANY SERIOUS INCIDENT HAS OCCURRED IN RELATION TO THE DEVICE, REPORT TO BREG AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED.

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