INDICATIONS: Indicated for post-injury or post-operative application to grade 1, 2, or 3 ankle sprains, and stable or internally fixed fractures of the foot or ankle. Also indicated for treatment of stable or internally fixed fractures or fractures with bridging callous of the middle to distal tibia or fibula.

CONTRAINDICATIONS: Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.

WARNING: United States Federal Law Restricts this Device to Sale by or on the Order of a Licensed Health Care Practitioner.

WARNING: For Single Patient Use Only.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device will not prevent or reduce all injuries.

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CAUTION: Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time.

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ENGLISH

1 Apply Calf Wrap

Slip the foot into the bootie with the toes under the toe piece. Close the forefoot cover over the foot. Trim the excess leaving 2"-3" (5cm-7.6cm) overlap.

2 Wrap the calf section around the leg, and trim the excess leaving 2"-3" (5cm-7.6cm) overlap. Close the hook and loop fastener at the front. Remove the forefoot strap from the uprights. Pull center of strap to loosen. **Note:** Do not pull the strap out of the slots on the side of the boot.

3 Slip Foot Into Boot

Slip the foot into the boot with the plastic covers over the uprights to prevent the hook and loop engagement.

4 Position the uprights in line with the leg and remove the plastic covers to engage the hook and loop fastener.

5 Fasten Straps

Pull the ends of the forefoot strap to tighten. Cross each end of the strap over the forefoot and attach to the hook and loop fastener on the opposite uprights.

6 Starting at the ankle, wrap the strap ends securely forward around the sides of the leg to interlock the strap with the uprights **Note:** Bottom strap overlaps part of the forefoot strap to help secure the forefoot strap in place. Repeat this step for remaining strap.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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