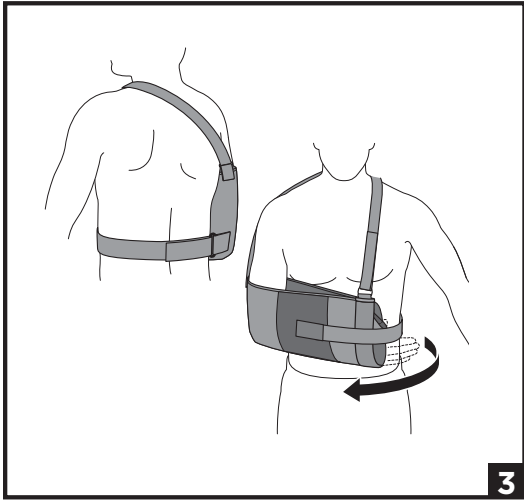
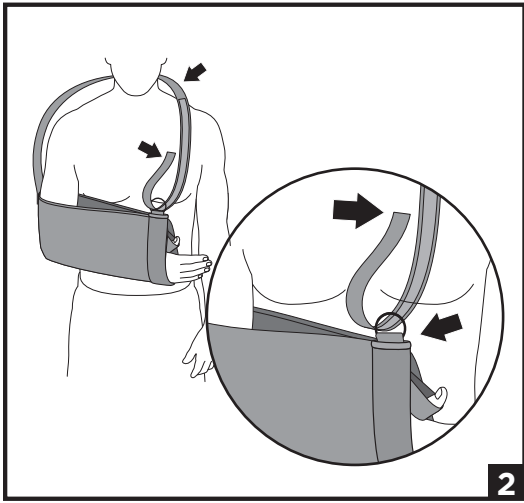
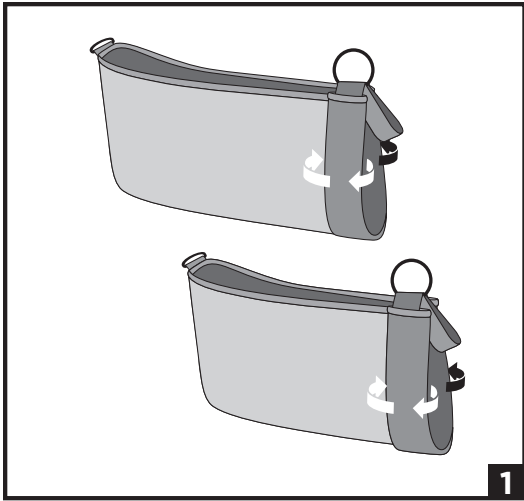
ENGLISH

INTENDED USE/INDICATIONS:

Indicated for dislocations, clavicle and humerus fractures.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

INSTRUCTIONS FOR USE:

1. To adjust the size of the universal sling, roll the cuff at the front of the sling to the correct arm length. Slide arm into the sling.
2. Wrap shoulder strap around neck and shoulders. Lace strap end through D-ring and attach to shoulder strap.
3. **Shoulder Immobilizer:** After fitting the sling as indicated above, feed waist strap through plastic ring at the back. Wrap snugly around the waist and secure the strap to the hook/loop patch on the front of the sling.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Acetal, Cotton, Nylon, Polyester, Polypropylene, Polyurethane

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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