**INDICATIONS FOR USE WITH THE ARC 2.0 WITH PILLOW:** For non-surgical or post-operative support and immobilization of the injured shoulder when varying degrees of internal rotation, external rotation and abduction are required. Positions include gunslinger, neutral plane, and statue of liberty.

**CAUTION:** This device is not intended for reuse on a second patient or for extended use beyond 180 days.

**CAUTION:** Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use

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and seek medical attention.

**CAUTION:** This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

**CAUTION:** This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

**CAUTION:** Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time.

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## Shoulder Immobilizer Conversion Kit

### INSTRUCTIONS FOR USE

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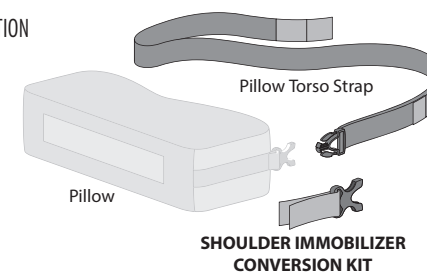
#### INSTRUKTIONER

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2382 Faraday Avenue, Suite 300  
Carlsbad, CA 92008 USA  
Toll Free Tel: 1-800-321-0607  
Local: +1-760-795-5440  
Fax: +1-760-795-5295  
www.Breg.com

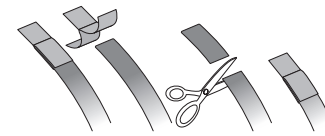
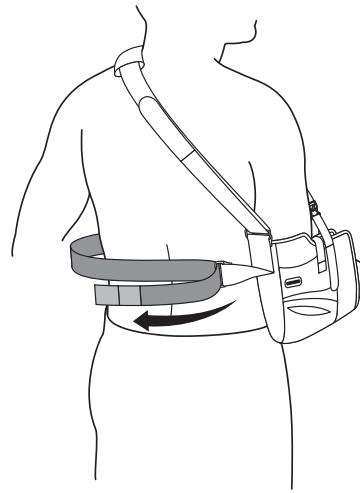
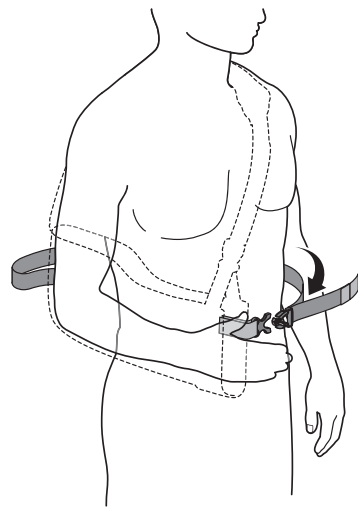
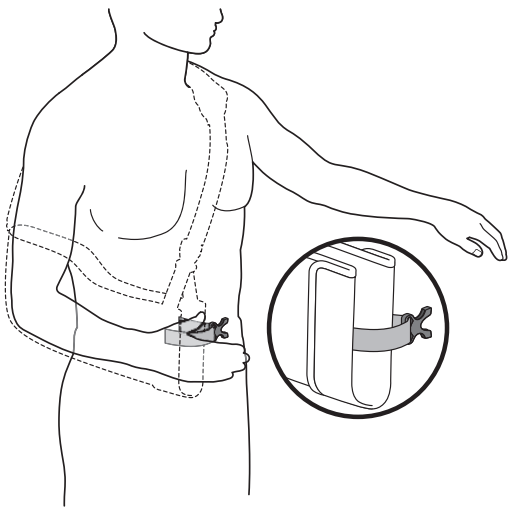
© 2023 Breg, Inc. All rights reserved.

AW-CP020272 Rev C 12/23

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany

## English

### APPLYING THE SHOULDER IMMOBILIZER CONVERSION KIT

Example: Right Shoulder

Attach the hook alligator tab to the inside of the arm sling (as shown) and insert the buckle at the front of the arm sling. Pass the torso strap around the opposite side of the body and through the extra D-ring at the back end of the arm sling. Loosen the hook closures at either end of the strap and pull the strap ends, front and back, to a comfortable tension. Refasten the closures.

**NOTE:** Excess strap may be trimmed if necessary.

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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