# Polar Care Y-Adapter

POLAR CARE Y-ADAPTER

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CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.

WARNING: A LICENSED HEALTH CARE PRACTITIONER MUST SELECT THE CORRECT TEMPERATURE SETTING. PATIENTS VARY IN SENSITIVITY TO COLD. A REGULAR CHECK OF THE TEMPERATURE MUST BE MADE ONCE IT HAS BEEN ESTABLISHED FOR THE PATIENT. CAUTION SHOULD BE TAKEN DURING PROLONGED USE, FOR CHILDREN, DIABETICS, INCAPACITATED PATIENTS, AND THOSE WITH DECREASED SKIN SENSITIVITY OR POOR CIRCULATION.

WARNING: DO NOT ATTEMPT TO STERILIZE THIS DEVICE BY ANY MEANS.

## POLAR CARE Y-ADAPTER

This Y-adapter will allow two BREG cold therapy pads to be applied simultaneously, in combination with a Polar Care 500, for external applications of cold therapy.

**NOTE:** Circulating device must circulate enough water to feed both pads. Check both pads to ensure they are cooling down upon initial start-up (i.e. no kinks or obstructions).

Follow the operating instructions for the pads and the pumping system for proper operation and application.

### APPLICATION INSTRUCTIONS:

- 1 Connect the male coupling of the Polar Care Y-adapter to the Polar Care 500 pump.
- 2 Connect the desired Polar Care Pad to the female coupling of the Y-adapter.
- 3 Ensure pads are cooling properly upon start-up.

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