## ⚠️ WARNINGS ⚠️

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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# Soft Hinged Knee Front Closure Instructions for Use

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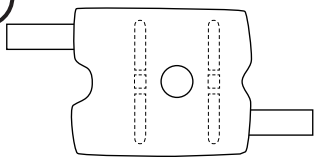
1

### 1. Loosen All Straps

Loosen all straps to their maximum circumference to open the brace.

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### NOTICE TO USERS

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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Material: Neoprene or AirMesh®

### CARE & WASHING INSTRUCTIONS

Hand wash in cold water with a mild detergent, rinse and lay flat to air dry.

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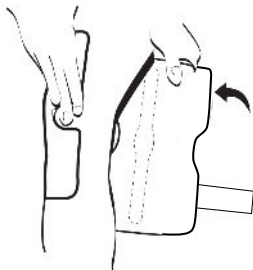
2

### 2. Apply Sleeve

Wrap the sleeve around the leg so that the hinges line up with the center of the knee. Secure the sleeve with the hook and loop fastener. If necessary, reposition the sleeve for an even fit. The front hole should be positioned over the knee cap.

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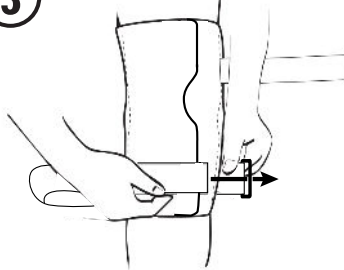
3

### 3. Secure Lower Strap

Wrap lower strap around the brace, then loop strap through D-ring. Pull strap back to tighten and press tab end to secure.

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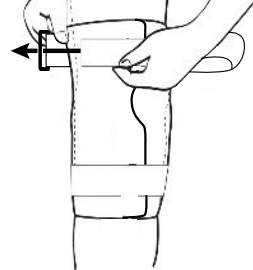
4

### 4. Secure Upper Strap

Wrap upper strap around the brace, then loop strap through D-ring. Pull strap back to tighten and press tab end to secure.

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©2023 Breg, Inc. All rights reserved.