DANSK

TILSIGTET BRUG/INDIKATIONER:

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ENGLISH

INTENDED USE/INDICATIONS:

Soft knee bracing products are designed to provide control and support for ligament injuries and instabilities.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient’s caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

INSTRUCTIONS FOR USE:

1. Loosen All Straps

Loosen all straps to their maximum circumference to open the brace.

2. Apply Sleeve

Wrap the sleeve around the leg so that the hinges line up with the center of the knee. Secure the sleeve with the hook and loop fastener. If necessary, reposition the sleeve for an even fit. The front hole should be positioned over the knee cap.

3. Secure Lower Strap

Wrap lower strap around the brace, then loop strap through D-ring. Pull strap back to tighten and press tab end to secure.

4. Secure Upper Strap

Wrap upper strap around the brace, then loop strap through D-ring. Pull strap back to tighten and press tab end to secure.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

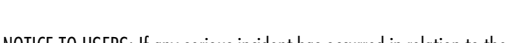
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
Aluminum, AirMesh®, Neoprene, Nylon, Polyester, Stainless Steel

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com





DISPOSAL:

Dispose of product according to local regulations.

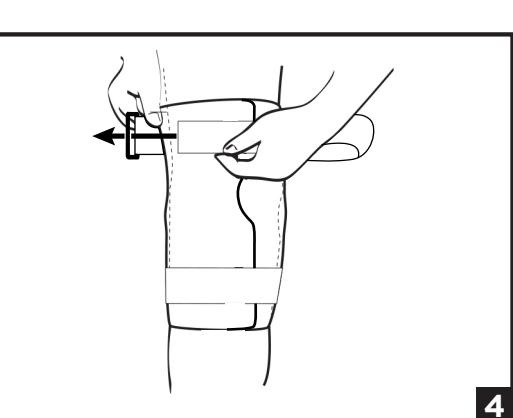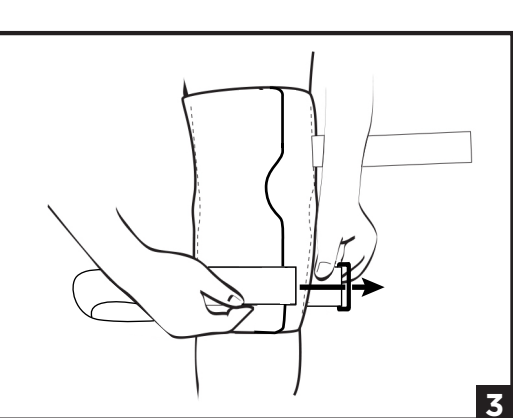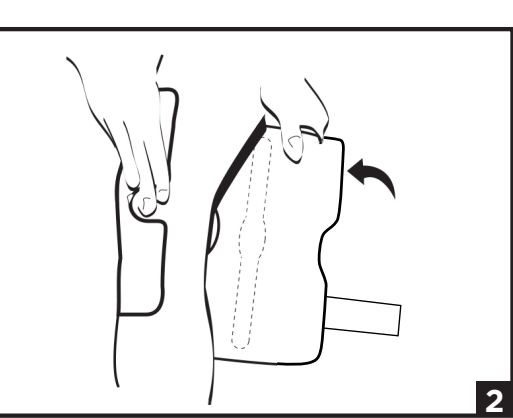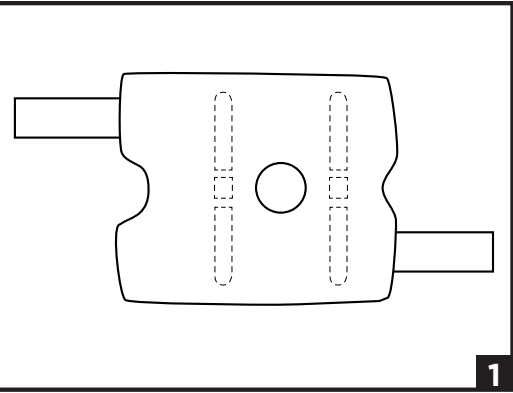NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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