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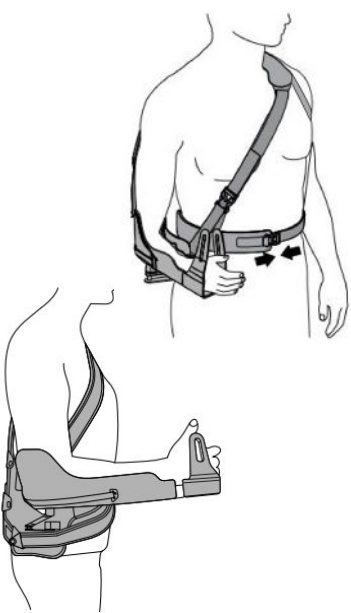
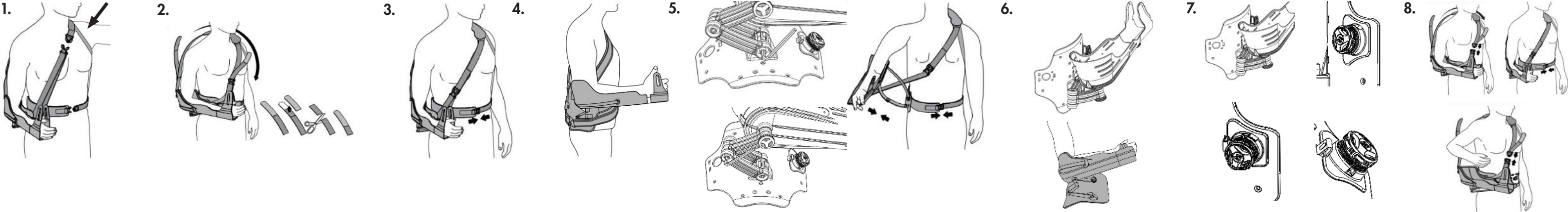
11.000032 Rev B 0824

English

INDICATIONS: For non-surgical or post-operative stabilization, immobilization, and mobilization of the injured shoulder when varying degrees of internal rotation, external rotation and abduction are required. For treatment of the stiff shoulder to assist in improving range-of-motion. Positions include gunslinger, neutral plane, and statue of liberty.

WARNING: For Single Patient Use Only.

CAUTION: This device is not intended for reuse on a second patient or for extended use beyond 180 days.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

CAUTION: This device will not prevent or reduce all injuries. **CAUTION:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.

CAUTION: This device has been developed for the indications listed above and may be damaged by unusual activities such as sports, falls, or other accidents, or exposure of the hinges to debris or liquids. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time.

CAUTION: Do not increase the reel knob tension past a position of comfort.

INSTRUCTIONS:

- Put forearm into cradle and slip non-affected arm through shoulder strap ring.
- Attach #1 green clip to green buckle.
- Adjust shoulder straps for sizing.
- Attach #2 yellow and #3 red clips to matching buckles. Adjust waist straps for sizing.
- Adjust the hand grip length.
- Adjust the abduction angle with the hex key (located under the cradle) and screws under the link bars.
- Adjust the rotation of the cradle as needed. Clockwise to tighten and counterclockwise to loosen.
- Engage the dynamic tensioning mechanism as prescribed by your physician.
- Turn the circular reel clockwise to engage and create tension. Turn counterclockwise to release.
- To remove the brace, support the bottom of the cradle and unclip the green buckle.
- Next, unclip the #2 yellow and #3 red waist buckles. Slide the arm out of the cradle with support from the non affected arm.

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© 2024 Breg, Inc. All rights reserved.