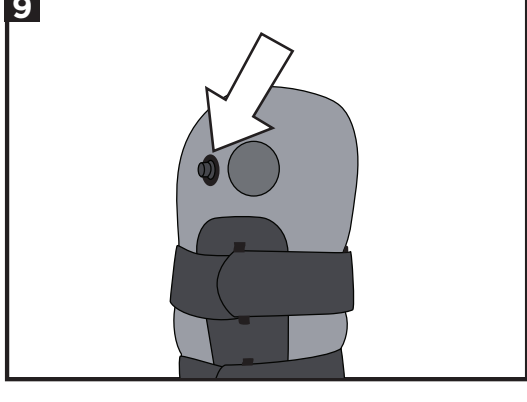
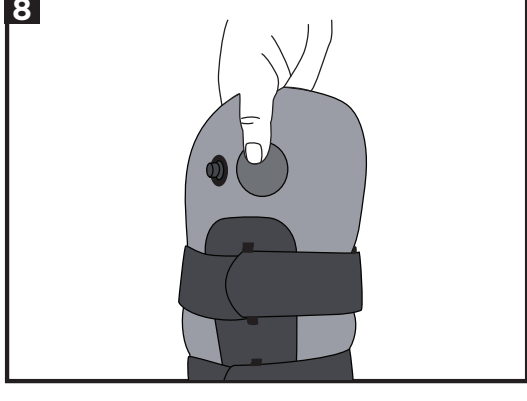
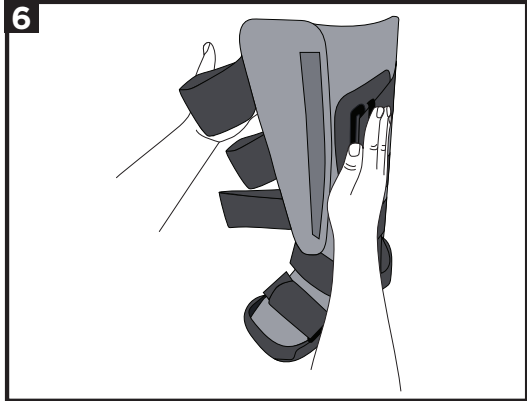
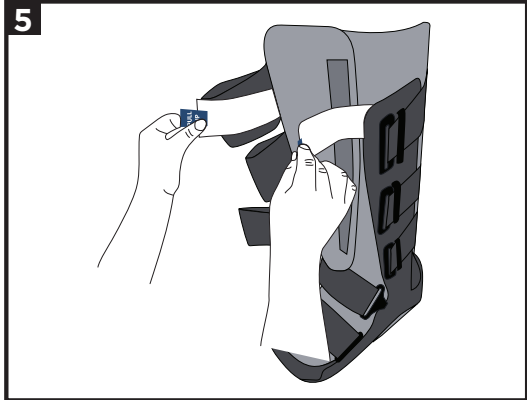
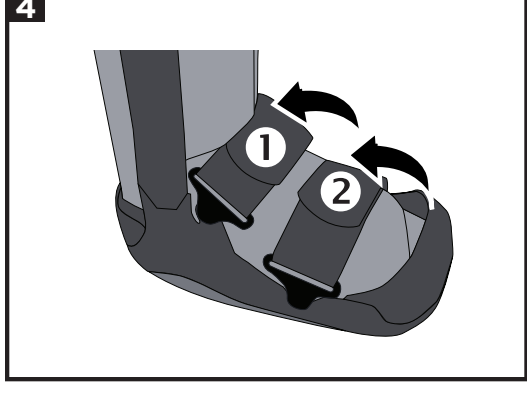
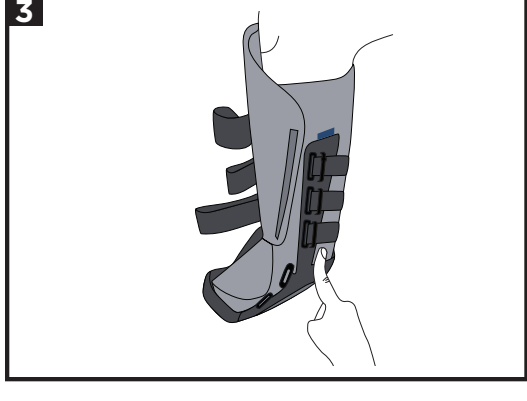
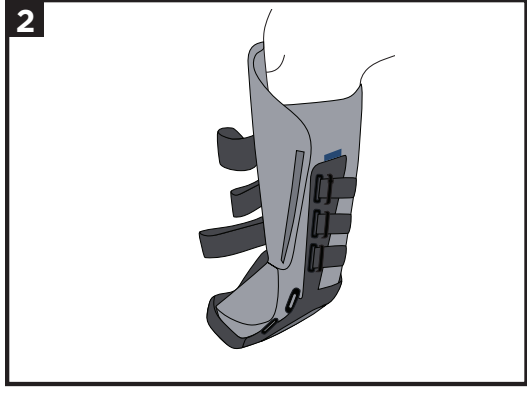
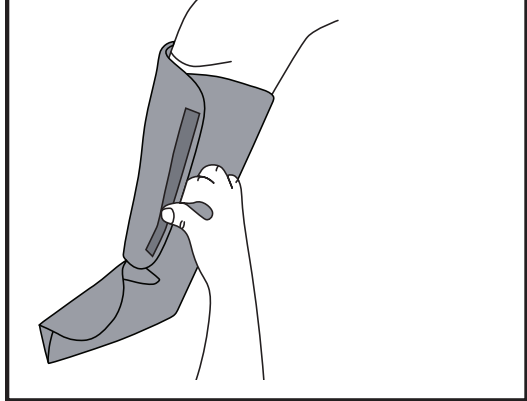
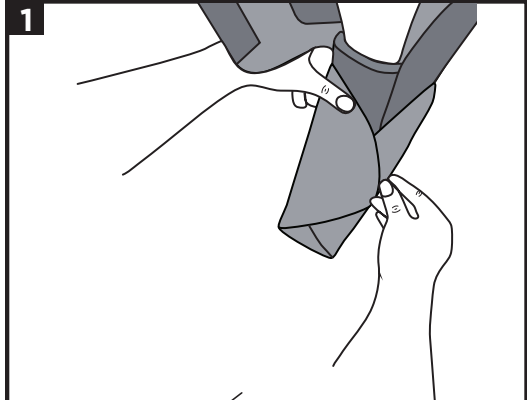
ENGLISH
INDICATIONS FOR USE
Foot and ankle products provide overall support, compression and/or protection to the lower leg including ankle, and act after an injury or surgery.

INTENDED USER
The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS

- This device is not suitable for unstable fractures or fractures of the tibia or fibula.
- Application of this device is recommended only when the fracture is demonstrably stable and there are acceptable limits of angular and rotational deformity.

WARNINGS & PRECAUTIONS

- CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS BEFORE USE.
- DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.
- THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.
- IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- IF YOU EXPERIENCE ANY SKIN IRRITATION OR ADVERSE SKIN REACTION WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR HEALTH CARE PROFESSIONAL.
- TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.
- DO NOT OPERATE A MOTOR VEHICLE OR HUMAN-POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.
- DO NOT OVERINFLATE THIS PRODUCT. SIGNS OF OVERINFLATION INCLUDE BUT ARE NOT LIMITED TO: DECREASED CIRCULATION (NUMBNESS AND TINGLING), SKIN SENSATION CHANGES, SKIN COLOR CHANGES, AND AN INCREASE IN PAIN.
- PLEASE CONSULT YOUR HEALTH CARE PROFESSIONAL BEFORE USING THIS PRODUCT ABOARD AN AIRPLANE AS THE INCREASE IN AIR PRESSURE CAN ADVERSELY AFFECT ITS NORMAL FUNCTION AND CAUSE FURTHER INJURY.
- DEVICES MAY NEED TO BE REMOVED PERIODICALLY.
- DO NOT REMOVE APPLIED WALKER BOOT UNLESS DIRECTED BY A MEDICAL PROFESSIONAL.
- FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES.
- THIS DEVICE IS NOT SUITABLE FOR UNSTABLE FRACTURES OR FRACTURES OF THE TIBIA OR FIBULA.
- DO NOT ALLOW DIRECT CONTACT OF THE PRODUCT WITH INJURED SKIN.
- DUE TO THE THICK SOLE OF THE BOOT, IT MAY LEAD TO A LEG LENGTH DISCREPANCY RESULTING IN SECONDARY PAIN IN THE BACK, HIPS, AND KNEES.
- WALKING WITH A LIMB IMBALANCE MAY CAUSE THE ENTIRE BODY TO BE MISALIGNED, LEADING TO ABNORMAL GAIT AND POOR POSTURE.

INSTRUCTIONS FOR USE

1. Apply liner to leg snugly.
2. Slide the walker onto the leg with liner. Ensure foot sits flat at the bottom and heel is pushed back.
3. Center walker uprights over malleoli (ankle bones).
4. Fasten straps snugly starting with toe straps.
5. Remove quick pulls from straps. Discard.
6. Attach uprights to liner pressing firmly.
7. Fasten the remaining straps.
8. Inflate air bladders to desired pressure by pressing the bulb.
9. Release air by opening cap next to pump and push down on release valve.

OPTIONAL APPLICATION INSTRUCTIONS

A. Heel removal: To remove the heel, first remove the liner from the boot. Remove the heel pad and insole. Locate the heel clip and pull up.

B. Heel clip installation: Lift heel pad and insole. Slide the heel clip into place using the star guides. Place the insole and pad in the proper position.

C. Adjust toe guards: Unstrap the number 1 strap first and rotate the Drings back. i. Lift the toe guard up and align. Align the hole lines with the drive tail slot guide, press on the sides, and lift to remove the toe guard. ii. To add, press on the sides of the toe guard and align with the slot, clip in, and rotate to the front.

D. Hook-Loop Patches: For additional support position hook-loop patches between the boot upright and the liner on either side, ensure the upright is centered over the patch. Adjust if necessary.

USE AND CARE
Hand-wash the liner and straps with neutral detergent in water below 30°C (86°F), rinse thoroughly, and air dry. Sterilization is not necessary.

MATERIAL CONTENT
Polypropylene, Nylon, Thermoplastic Rubber, Aluminum, Polyester, Polyurethane.

WARRANTY
Breg, Inc.'s obligation under this warranty is limited to the replacement or repair of any defective part or parts of this product. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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