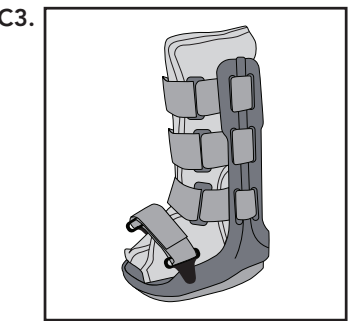
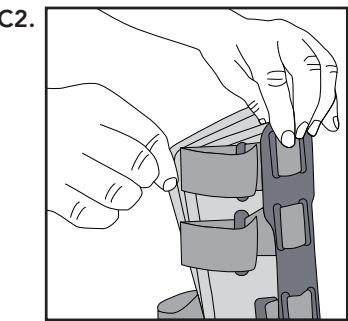
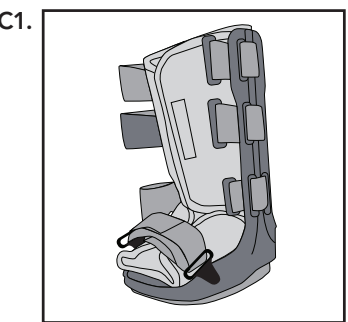
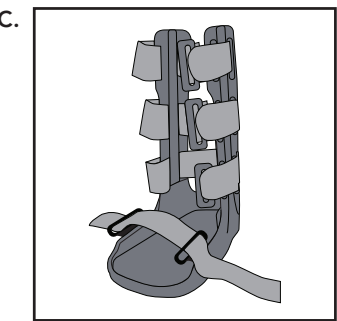
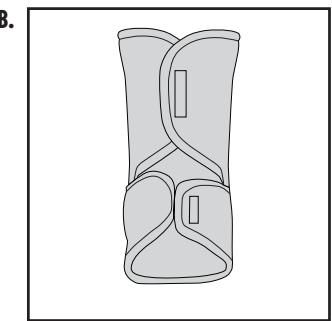
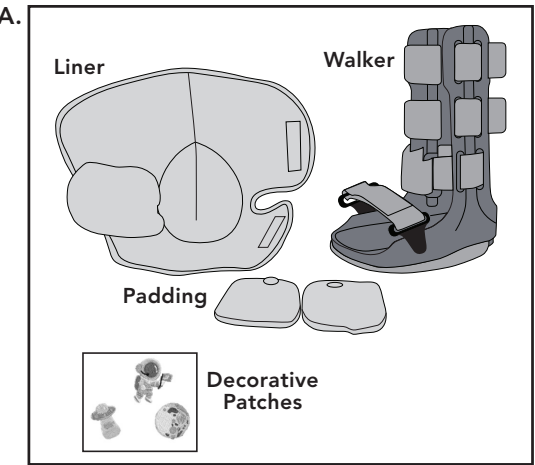
Svenska

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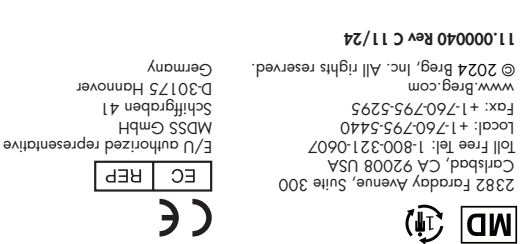
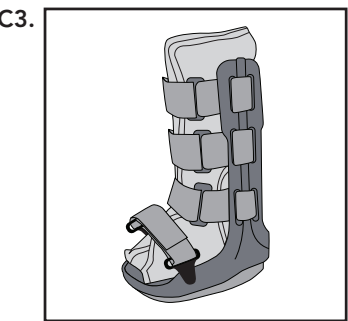
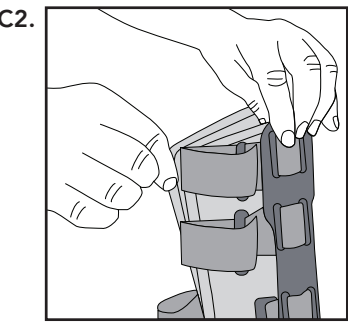
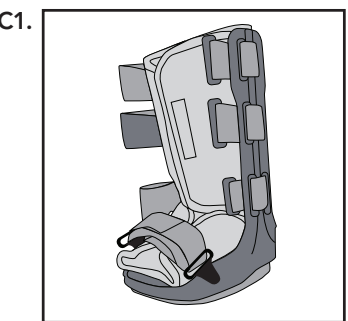
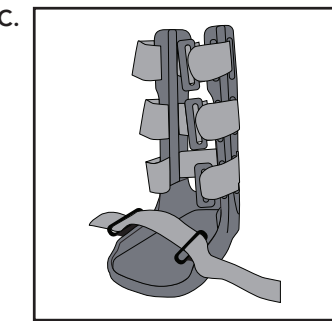
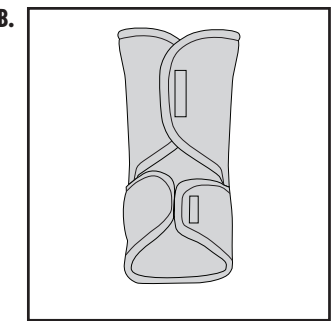
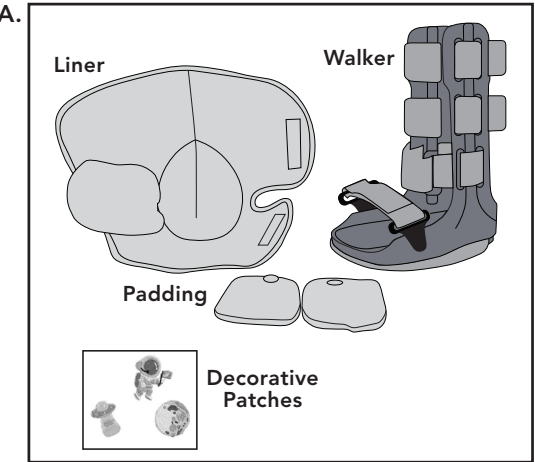
English

Intended Use:

The device is used for the treatment of foot fractures and contusions. It is intended for single-patient use.

Intended Users:

Children from 2 year old to 7 years old. Please refer to sizing chart with the QR Code.

Application Instructions:

A. Parts included: Liner, walker, decoration patches and padding for proper fit.

B. Remove liner from boot, and place the injured foot into liner.

C. Place foot w/liner into walker with heel seated to back of boot. Starting at the toe strap, secure all straps on boot. Ensure straps are snug, but not too tight.

D. If you have a air version. Inflate air bladders to desired pressure by pressing the bulb; and release air by opening cap next to pump and push down on release valve.

E. (Optional) Add decorative patches as wanted.

Care Instructions:

Hand wash the liner and straps with mild soap, air dry.

Warnings/Contraindications:

WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS BEFORE USE.

WARNING: DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.

WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

WARNING: IF YOU EXPERIENCE ANY SKIN IRRITATION OR ADVERSE SKIN REACTION WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR HEALTH CARE PROFESSIONAL.

WARNING: TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.

WARNING: DO NOT REMOVE APPLIED WALKER BOOT UNLESS DIRECTED BY A MEDICAL PROFESSIONAL.

WARNING: CHOCKING HAZARD - SMALL PARTS.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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