ENGLISH

INDICATIONS FOR USE:

Breg's Polar Care® is intended for application of external cold therapy for temporary reduction of pain, swelling, and/or inflammation resulting from injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

WARNINGS AND PRECAUTIONS:

- Non-sterile. Do not use directly over breached skin.
- For single patient use only.
- Do not attempt to sterilize this device by any means.
- Do not puncture the pad.
- Carefully read use instructions and warnings provided with Polar Pad and associated Polar Care device.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- Follow the licensing health care practitioner's protocol regarding method, frequency and duration of administration.

INSTRUCTIONS FOR USE:

- 1.** Peel and stick the squares of hook and loop to the smooth side of a Polar Pad (evenly spaced).
- 2.** Attach the Polar Pad to the inside of the Polar Wrap. Position the pad to cover the traumatized area of the body when the wrap is applied to the patient.
- 3.** Secure the wrap with the pad to the patient.
- 4.** Trim the wrap to fit if needed.

USE AND CARE:

- Hang the pad upside down to drain excess water into drain or sink.
- Hand wash the outside of the pad as needed with a soft cloth, mild soap and 30°C (86°F) water. Rinse thoroughly.
- Wipe dry before storing.
- DO NOT machine wash or tumble dry.

MATERIAL CONTENT:

Nylon, Polyurethane

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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Polar Care is a registered trademark of Breg, Inc.
© 2025 Breg, Inc. All rights reserved.