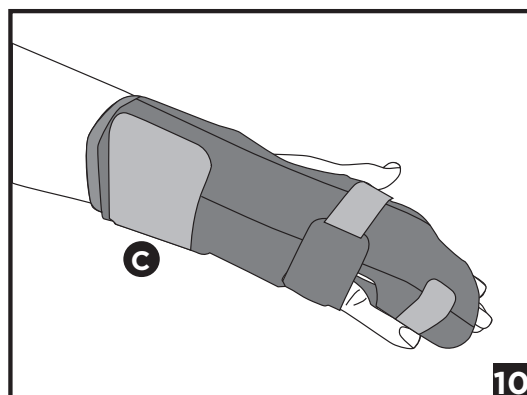
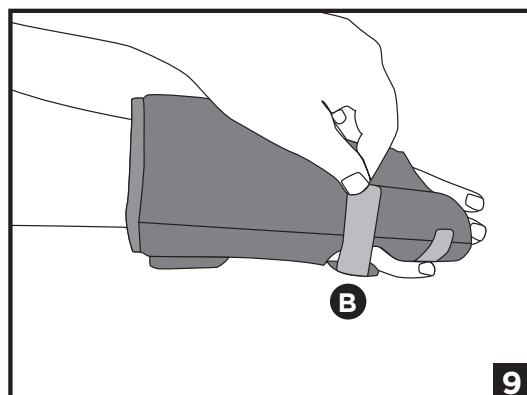
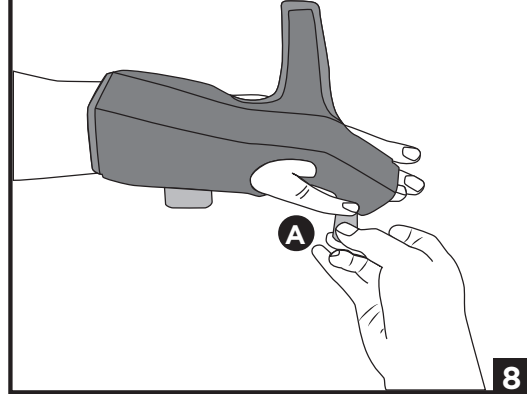
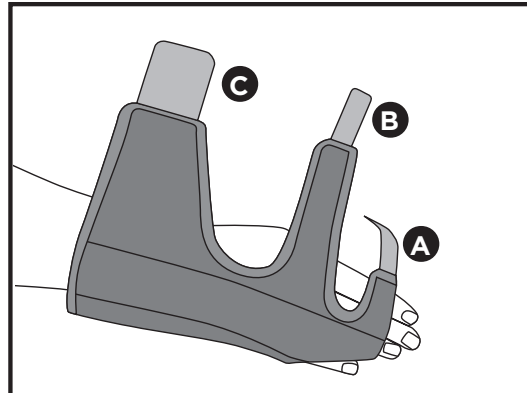
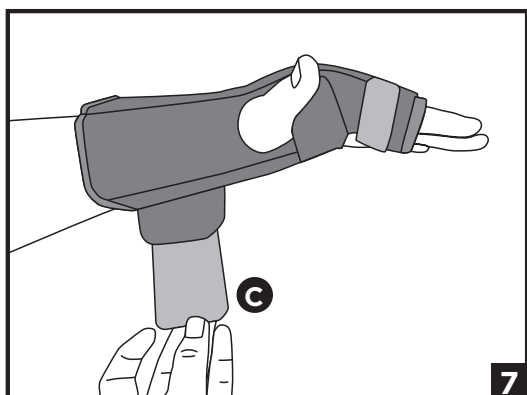
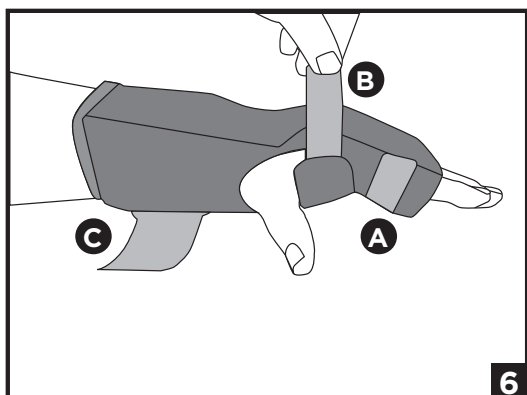
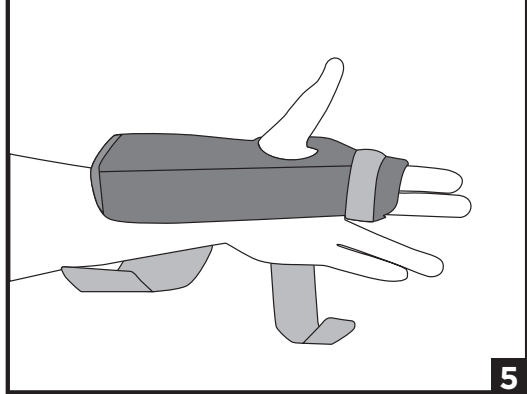
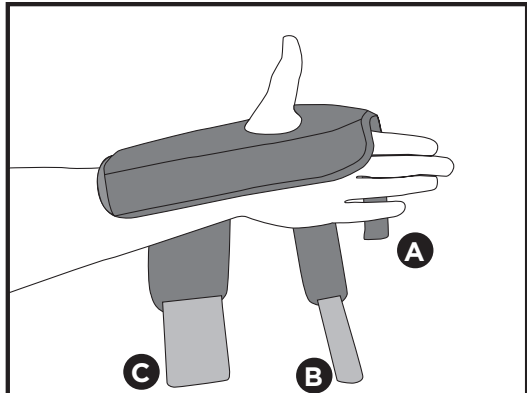
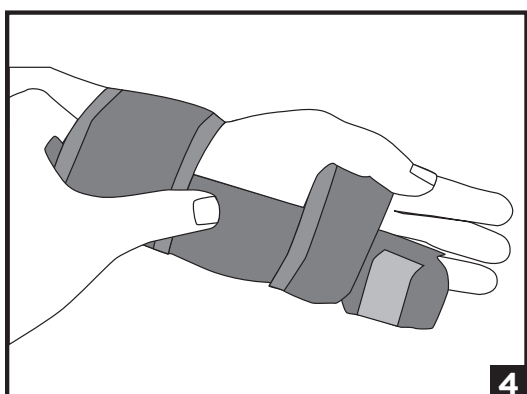
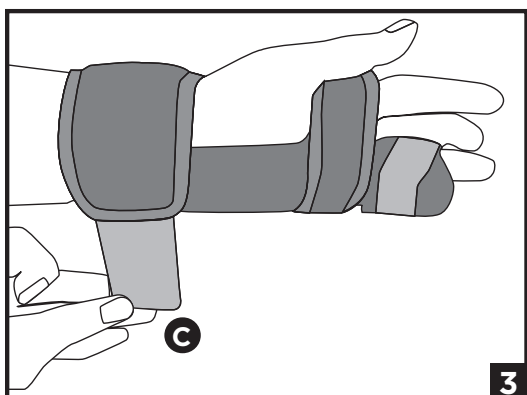
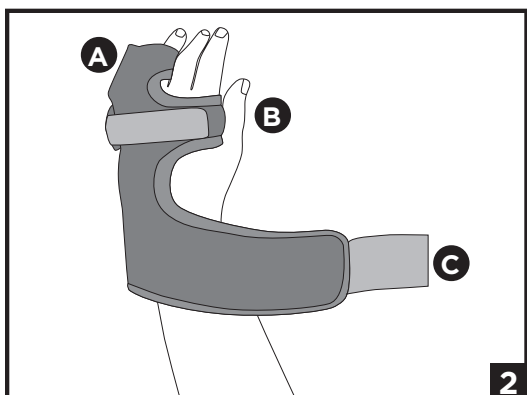
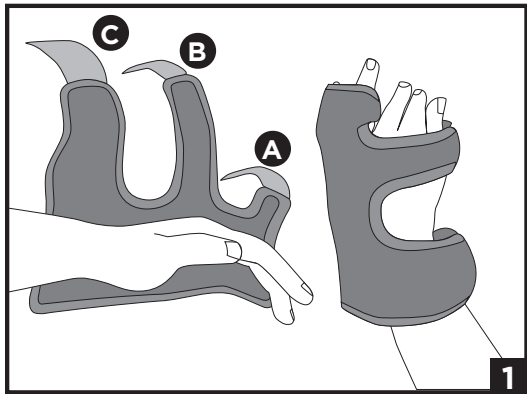
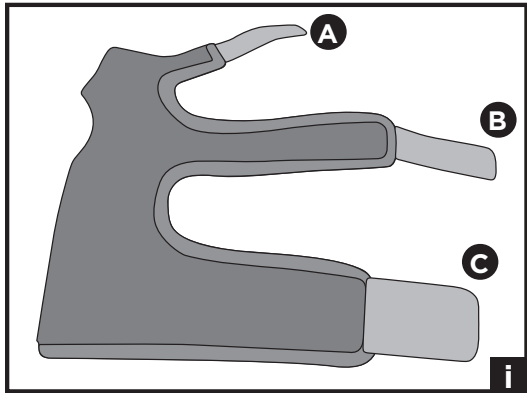
## ENGLISH

### INTENDED USE/INDICATIONS:

Intended to provide stabilization and immobilization of fractures or injuries of the metacarpals, proximal phalanges, and their joints. Indicated for Boxer's Fracture, Radial Gutter, and Metacarpal Fractures.

### INTENDED USERS:

The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

None.

### ⚠ WARNINGS AND PRECAUTIONS:

- For single patient use only. Do not reuse for multiple patients. Cross contamination may occur if shared.
- Do not use this device if it was damaged.
- Do not modify the device or use the device other than intended. Any unauthorized modifications, part substitutions, or reassembly may result in device failure or injury.
- The frequency and duration of use should be determined by a healthcare professional.
- Discontinue immediately if persistent redness, swelling, pain, irritation, or other unusual symptoms occur.

### INSTRUCTIONS FOR USE:

Knockout can be applied to treat three indications. Please ensure you are using the correct application for your specific need. For all applications, refer to straps **A**, **B**, and **C** as illustrated in **Figure 1**.

Note: Ensure all straps are secure without overtightening.

#### BOXER'S FRACTURE APPLICATION

- Position the brace on the palmar side of the hand and wrist, ensuring it contours naturally to the fingers and hand. Then, wrap strap **A** around the ring and little fingers.
- Fasten strap **B** around the palm.
- Fasten strap **C** around the wrist.
- Optional:** To relieve pressure on the ulnar styloid, the dorsal stay may be contoured as needed.

#### RADIAL GUTTER APPLICATION

Note: This application uses a right-hand brace to treat a left-hand injury, and a left-hand brace to treat a right-hand injury.

- Place the brace on the volar surface of the hand and wrist, sliding the thumb through the designated opening. Adjust the hand to ensure the brace aligns with the natural contours of the fingers and palm. Wrap strap **A** around the index and middle fingers.
- Fasten strap **B** around the palm.
- Fasten strap **C** around the wrist.

#### METACARPAL FRACTURE APPLICATION

- Place the brace on the dorsal side of the hand and wrist. Slide the pinky finger through the designated opening. Wrap strap **A** around the middle and ring fingers.
- Fasten strap **B** around the palm.
- Fasten strap **C** around the wrist.
- Optional:** To relieve pressure on the ulnar styloid, the dorsal stay may be contoured as needed.

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

Note: If not rinsed thoroughly, residual soap can cause skin irritation or deterioration of the material.

### MATERIAL CONTENT:

Aluminum, Neoprene, Nylon, Polyester

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

## FRANÇAIS

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Stabilization and immobilization of fractures or the lesions of the metacarpals, the proximal phalanges and their articulations. Indicated for the fractures of the boxer, the gutter radial, and the metacarpal fractures.

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