ENGLISH

INDICATIONS FOR USE:

PodusGuard is intended to assist in the prevention and treatment of lower extremity conditions related to immobility or trauma, including pressure ulcers and ankle or foot contractures. It maintains the foot and ankle in a neutral position, offloads pressure from the heel, and supports proper limb alignment.

INTENDED USERS:

Intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠️ WARNINGS AND PRECAUTIONS:

- For single-patient use only. Do not reuse for multiple patients. Cross-contamination may occur if shared.
- Do not use this device if it was damaged.
- Do not modify the device or use the device other than as intended. Any unauthorized modifications, part substitutions, or reassembly may result in device failure or injury.
- Do not use over open wounds.
- Remove after prolonged wear to inspect skin integrity and monitor for signs of pressure injury.
- The frequency and duration of use should be determined by a healthcare professional.
- If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, discontinue use and consult your medical professional immediately.
- Do not use while showering or bathing.
- Clean product according to the provided washing instructions.

INSTRUCTIONS FOR USE:

- Open the straps and place the patient's foot into the boot
- Wrap the forefoot strap first (A), then the ankle strap (B), and finally the calf strap (C). Trim the remaining hook if needed.

ADJUST THE TOE EXTENSION:

- Remove the bottom sole, remove the toe extension by twisting the thumb-screw counter-clockwise using your fingers or a coin. Adjust the toe piece to the desired position and apply thumb-screw and tighten.

POSITION THE ANTI-ROTATION BAR:

- Loosen knob on the posterior side and position the anti-rotational bar as needed. Tighten the knob to secure the anti-rotation bar in the new position.

SOLE ATTACHMENT:

- Attach the sole by placing it on the bottom of the boot. The boot is used for limited ambulation. Remove the sole when the patient is in bed to prevent cross-contamination.

USE AND CARE:

Hand wash the liner in 30°C (86°F) water with mild soap, rinse thoroughly. Air dry only. Do not machine wash or tumble dry. Wipe the plastic shell with a damp cloth.

MATERIAL CONTENT:

Polyketone, PU Foam, Polyester/Acrylic Fiber, Polyester Fabric, Nylon, Polyester/Spandex Elastic, ABS, EVA, Steel, Brass

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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PodusGuard is a registered trademark of Breg, Inc.
© 2025 Breg, Inc. All rights reserved.