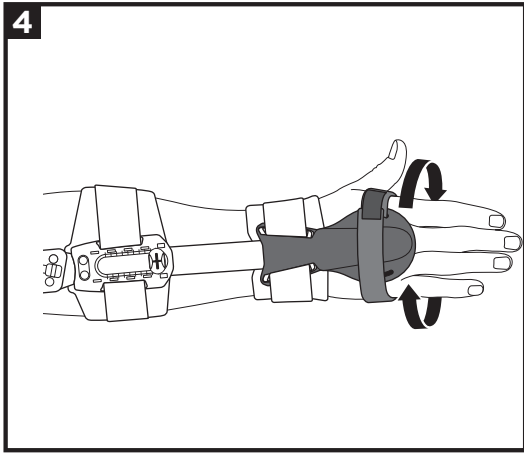
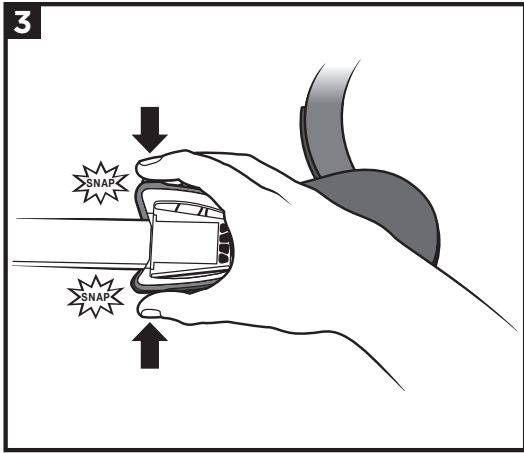
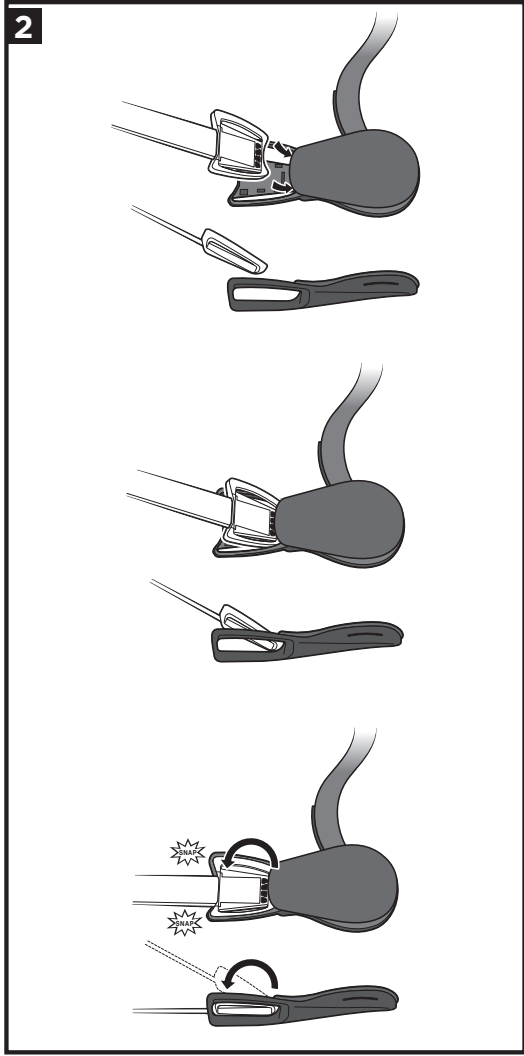
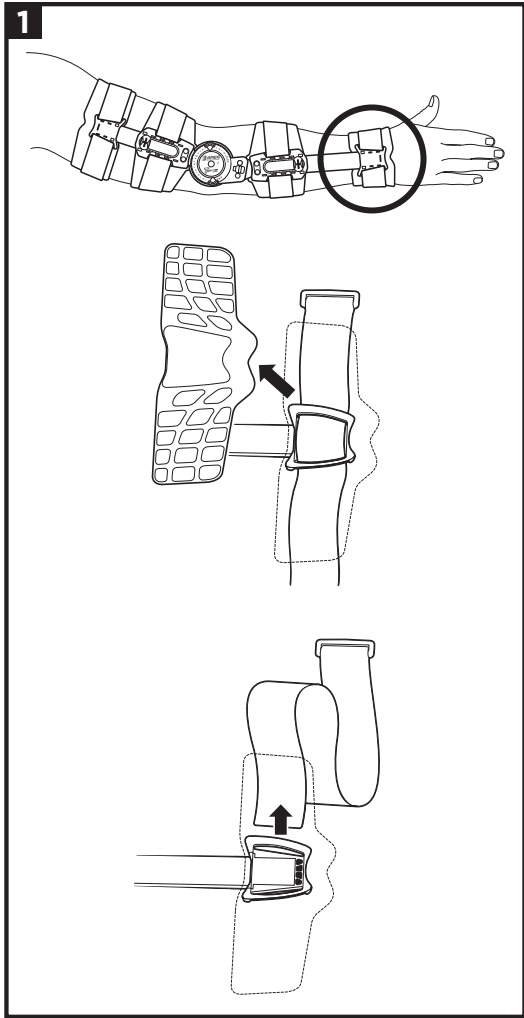
ENGLISH

INTENDED USE/INDICATIONS:

The intended use for Elbow bracing is to provide stability and range or motion control following elbow injury or surgery. The hand attachment provides range of motion restriction and hand pronation and supination control following injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read use/care instructions and warnings prior to use.
- Do not remove T Scope elbow brace unless instructed by your medical treatment professional. Do not change range of motion hinge settings without supervision by a medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your medical treatment professional regarding safe and appropriate activity level while wearing this device.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner.
- For single patient use only.

INSTRUCTIONS FOR USE:

1. The Neutral Hand Accessory should be attached to the end of the T Scope Elbow brace, closest to the hand. Start by removing the wrist pad and strap from the brace.
2. Next, lead the nose of the brace end into the Neutral Hand Accessory and "snap" together.
3. Ensure the accessory is locked to the brace by "pinching" the edges of the accessory to the brace with the thumb tabs. Then, replace the strap and wrist pad that were removed in step 1.
4. Finally, with the T-Scope Elbow brace properly applied to the affected arm, wrap the strap on the Neutral Hand Accessory around the hand as shown. The hand and wrist are now secured and pronation and supination will be reduced.
5. To remove the Neutral Hand Accessory, first remove the wrist pad and strap from the T-Scope Elbow brace. Next, depress both thumb clips to separate the proximal edges from the brace. Hold the Neutral Hand Accessory and the T-Scope Elbow brace and then collapse together separating the parts. Finally, replace the strap then wrist pad to the elbow brace.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Nylon, Polyethylene

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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