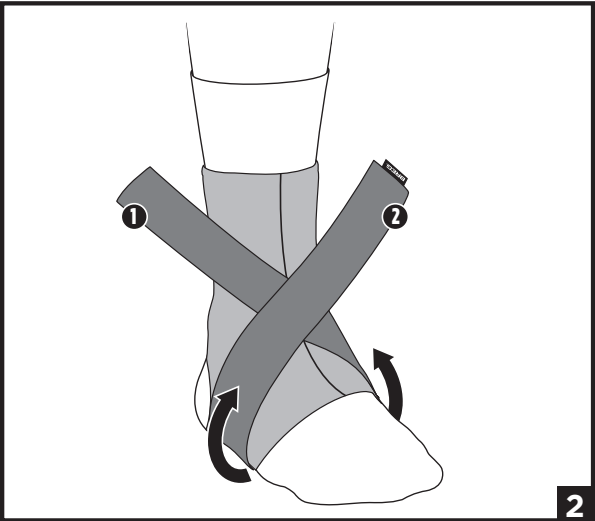
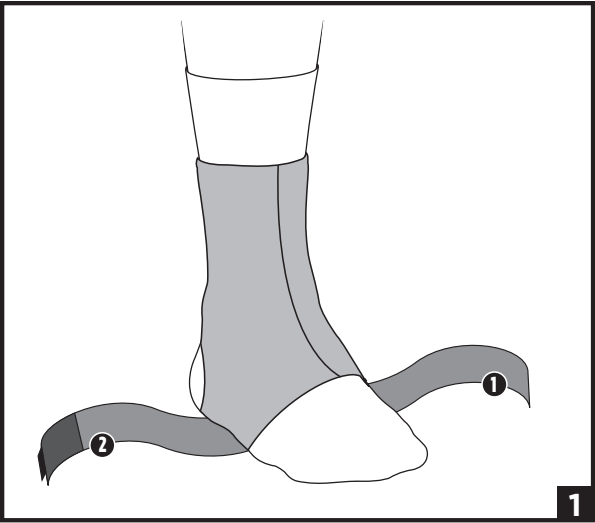
ENGLISH

INTENDED USE/INDICATIONS:

The intended use for ankle bracing products is to provide support and / or immobilization to the ankle for patients from ankle injury.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Follow application and care process described in instructions.
- For single patient use only.

INSTRUCTIONS FOR USE:

1. Slide sleeve onto foot. Position heel in the hole at the rear and bottom of the sleeve.
2. When the brace feels comfortable, cross the elastic straps over the top of the foot.
3. Continue tightly wrapping strap #1 in figure-8 design, and hold in place around the top of the ankle. Wrap strap #2 in figure-8 design, and secure tightly around the top of the ankle. Adjust for comfort.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Neoprene, Nylon, Polyester

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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