
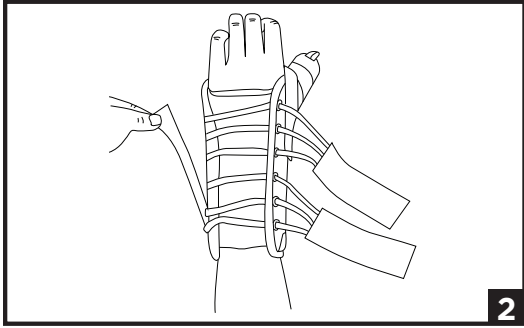
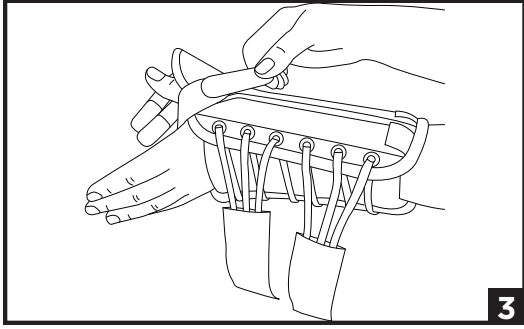
1


2

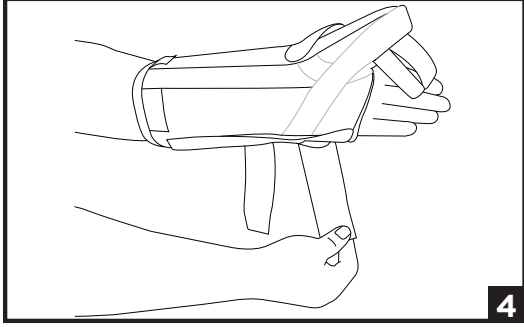
3

4

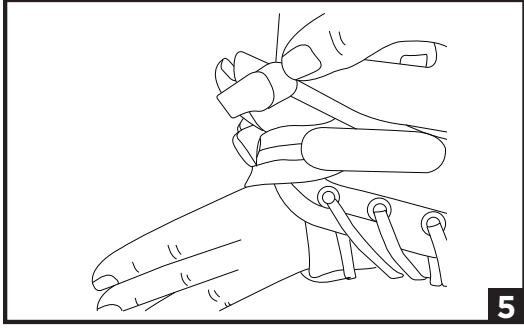5

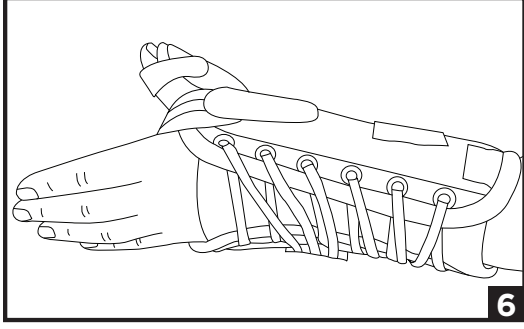6

#### ENGLISH

#### INTENDED USE/INDICATIONS:

The intended use wrist bracing products is to provide stability, support, and / or immobilization to the wrist joint for patients recovering from a wrist injury or surgery.

#### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient’s caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

#### CONTRAINDICATIONS:

Contraindicated for unstable fractures of the wrist.

#### ⚠️ WARNINGS AND PRECAUTIONS:

Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

- This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.
- This device has been developed for the indications listed above. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.
- Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C) for any extended period of time.
- This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for re-use on a second patient.

#### INSTRUCTIONS FOR USE:

##### 1. Position the Brace

Place hand in the brace making sure the palm stay is below the crease in the palm of the hand and the palmer pad is in the center of the palm.

##### 2. Adjust Ulnar Stay

The ulnar stay should be centered on the mid-line of the arm. To adjust, remove the ulnar stay and re-secure in desired position.

##### 3. Secure Hand

While maintaining the brace position, pull and secure the narrow strap between the thumb and pointer finger.

##### 4. Tighten Laces

Starting with the strap closest to the wrist, tighten laces by pulling on the strap end. Wrap and secure the strap. Repeat for the forearm strap.

##### 5. Secure Thumb

Pull and secure the two remaining straps around the thumb.

##### 6. Ensure Fit

Knuckles should not be obstructed, and brace should fit snug. Trim straps if necessary.

#### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

#### MATERIAL CONTENT:

Aluminum, Brass, Cotton, Nylon

#### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

#### DISPOSAL:

Dispose of product according to local regulations.

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