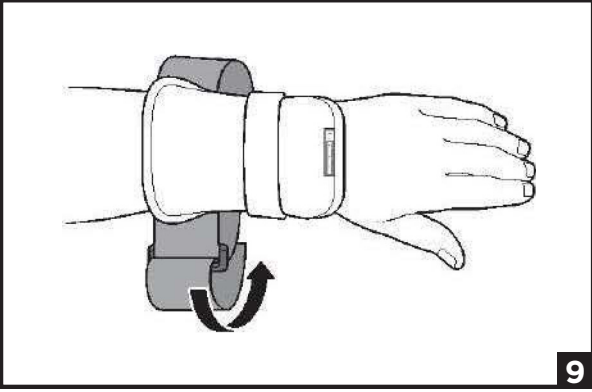
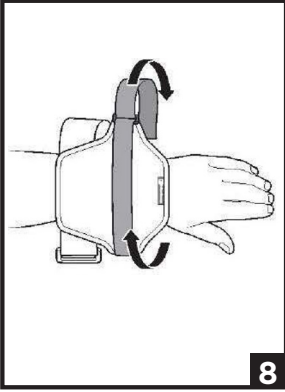
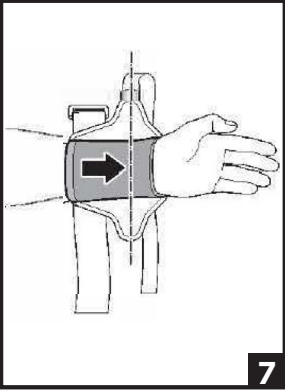
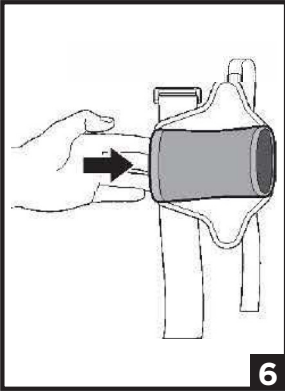
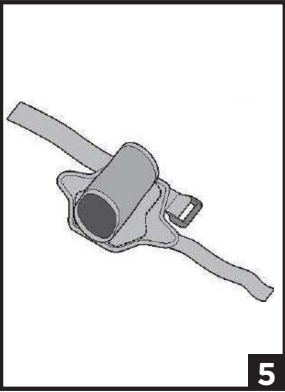
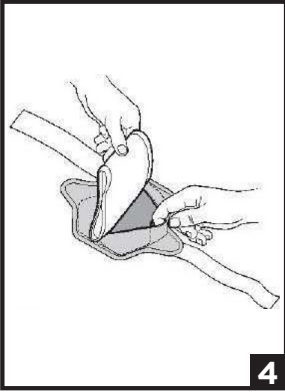
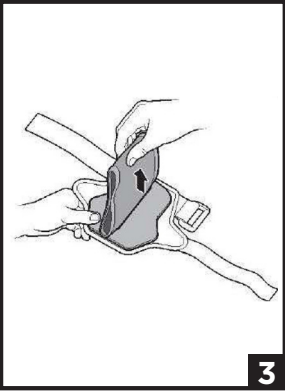
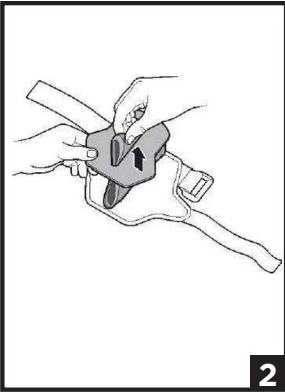
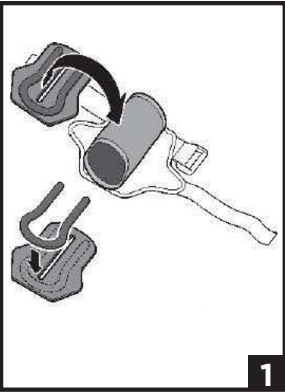
ENGLISH

INDICATIONS FOR USE:

The Wrist Guard is engineered for support when range of motion (ROM) needs to be controlled in one direction. The Wrist Guard is also for prophylactic use in sports and weightlifting where hyperextension often occurs. With adjustable inserts, users decide on ROM limitation, creating a custom fit depending on the level of stability needed.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions prior to use. To ensure proper performance, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.
- Single patient use.

INSTRUCTIONS FOR USE:

1. Attach the ROM horseshoe to the foam pad. Position the foam pad with horseshoe and Wrist Guard as shown.
2. Position the foam pad with the horseshoe facing down. Pull the Wrist Guard sleeve through the slot in the foam pad.
3. Pull the sleeve completely through the slot so the foam pad is against the Wrist Guard wrap.
4. Cover the foam pad with the Lycra® fabric on the inside of the Wrist Guard wrap.
5. The foam pad is properly positioned inside the Lycra fabric of the Wrist Guard wrap. The Wrist Guard is ready to apply to the wrist.
6. Slide your hand through the sleeve of the Wrist Guard.
7. The distal strap should be aligned with the wrist joint line as shown.
8. Secure the distal strap.
9. Secure the proximal strap and trim excess material from the straps if necessary.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Ethylene-vinyl acetate (EVA), Lycra, Neoprene, Nylon, Polyester.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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Lycra is a registered trademark of Breg, Inc.
© 2025 Breg, Inc. All rights reserved.