ENGLISH

Refer to the Instructions For Use (IFU) for device information: intended use, intended users, contraindications, and disposal.

⚠️ WARNINGS AND PRECAUTIONS:

- The Polar Care® Kodiak® can be cold enough to seriously injure the skin.
- Follow the Operating Instructions (on the back and side of the unit), Product Insert (in the pouch on side of unit), and the Intelli-Flo® Pad Fitting Instructions (provided with each Intelli-Flo Pad).

INSTRUCTIONS FOR USE:

This Y-adapter allows two BREG Intelli-Flo pads to be applied simultaneously, in combination with a Polar Care Kodiak, for cold therapy applications in multiple treatment areas.

NOTE: The Kodiak system requires enough water to feed both pads. The addition of water may be necessary after initial start-up, due to water displaced into the pads. Once both pads are filled, fill unit with additional ice and water as necessary.

1. Connect the center coupling of the Intelli-Flo Y-adapter to the Kodiak system.
2. Connect the remaining two couplings to each of the Intelli-Flo pads.
3. Ensure pads are cooling properly upon start-up. If necessary add additional ice and water to the Kodiak system.

USE AND CARE:

Warm water 30°C (86°F) and mild detergent may be used occasionally to clean the pump and tubes.

MATERIAL CONTENT:



ABS (Acrylonitrile Butadiene Styrene), EPDM (Ethylene Propylene Diene Monomer), Nitrile, PC (Polycarbonate), PP (Polypropylene), PVC (Polyvinyl Chloride), Stainless Steel

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

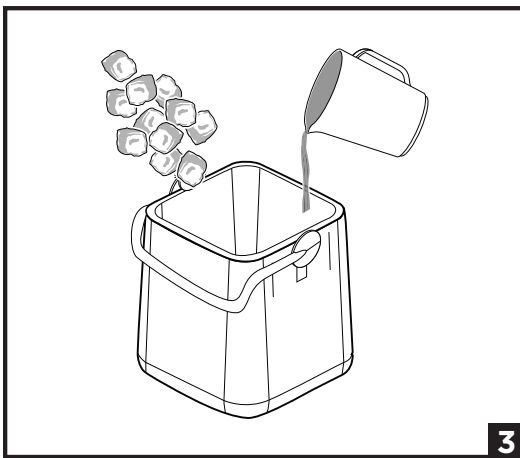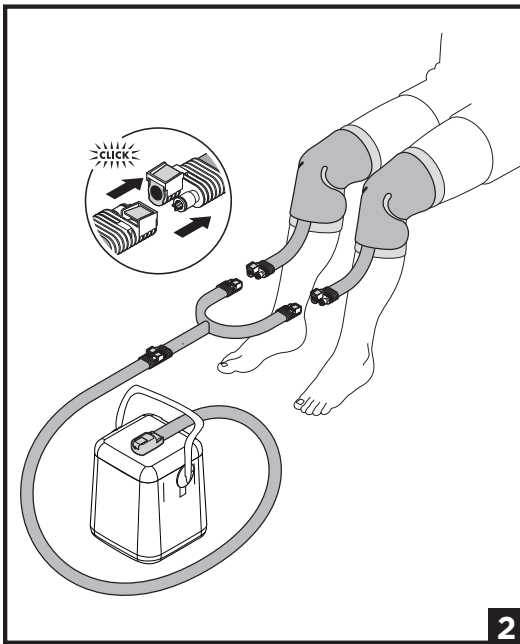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