
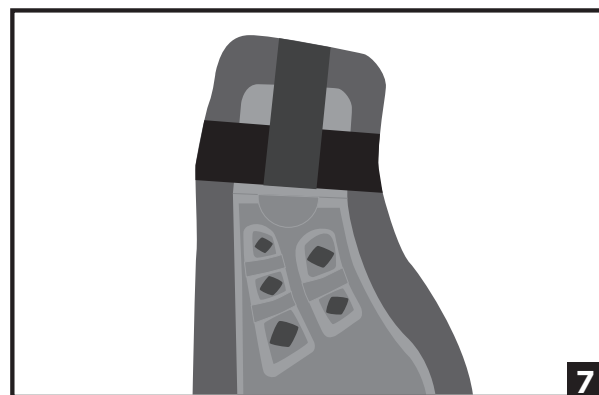
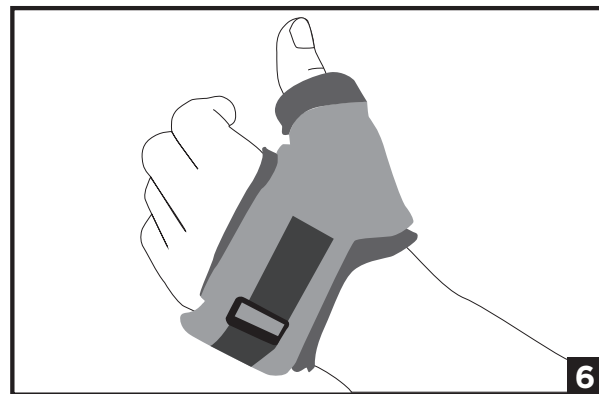
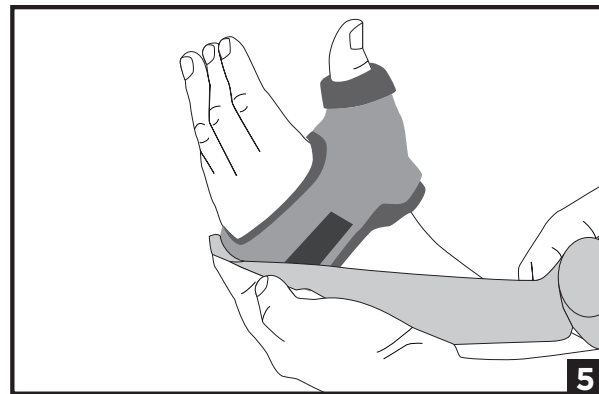
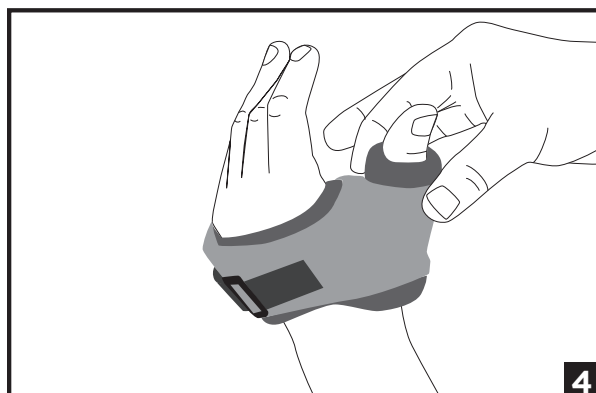
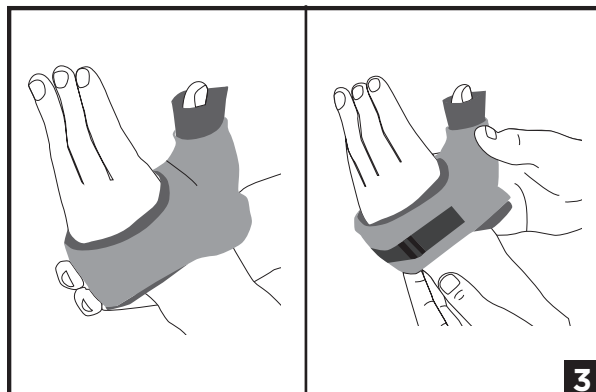
Cat.#	Measurement	Size	Qty
3401	235mm 9 1/4" 205mm 8"	Medium/Large M/L	6/box
3402	210mm 8 1/4" 175mm 6 3/4"	Small/Medium SM/M	6/box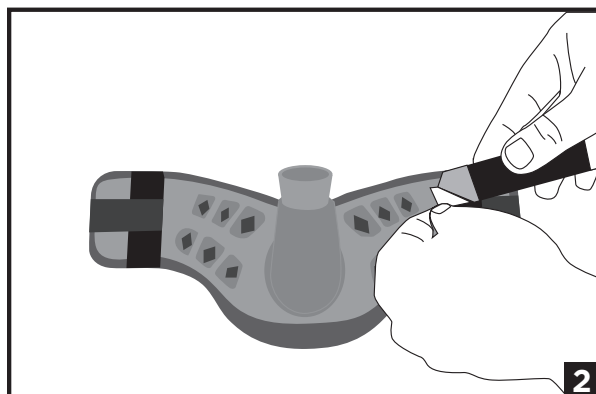

ENGLISH

INTENDED USE/INDICATIONS:

Treatment of ulnar collateral ligament injury or to provide support and immobilization in the treatment of soft tissue injuries. Immobilization and stabilization of the first MCP joint.

INTENDED USERS:

The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Simple, complex, compound and/or comminuted fractures. Not for treatment of an open wound.

⚠️ WARNINGS AND PRECAUTIONS:

- Device to be fitted by a qualified healthcare professional.
- This Device should not be used on a patient with polyester or elastane allergy.
- Ensure the orthosis is not too hot prior to application.
- Instruct the patient on correct adjustment of the strap to maintain snug fit.
- Remove the device before any re-molding with a heat-gun, to avoid burning the patient.
- The orthosis will lose its shape above 57°C/135°F or under extreme sunlight. Cover with stockinet or clothing.
- Avoid water or vapor above 40°C/104°F for prolonged periods. Avoid heat sources for long periods.
- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner. The frequency and duration of use shall be prescribed by a healthcare professional.

INSTRUCTIONS FOR USE:

1. Measure the hand with a tape measure across the palmar crease and select the correct size by using the measurement chart.
2. Remove the backing paper from the round end (A) of the strap and attach to the recess on the orthosis according to which hand to treat (R or L).
HEAT: Pre-heat the FastForm Primo Oven. Proper heating temperature is 260°F (127°C) in the Primo Oven.
Place in the FastForm Oven, flat, with the fabric side down and heat until it is soft and malleable.
NOTE: Heating time may vary (5-10 min) depending on whether you are using a FastForm oven or other Convection oven.
Primo Oven: 5 mins at 260°F (127°C)
Pro Plus/Elite Ovens: 5 mins at 220°F (105°C)
Preheated Convection Oven: 5-10 minutes at 167-185°F (75-85°C)
Do not heat more than one hour.
3. Apply the orthosis over the thumb and gently stretch the palmar wing across the palm and attach it to the back of the hand. Remove the backing paper and press the adhesive tab (C) into the soft Polytrex. Confirm proper alignment by making sure the top edge is below the 5th metacarpal and along the palmar crease.
4. Roll back the thumb liner (D) to form a soft edge.
5. Wrap a damp elastic bandage firmly but not tightly to form the orthoses. Mold the orthosis while keeping the thumb in the correct position.
6. Allow the orthoses to cool and harden then remove the elastic bandage.
7. Once orthosis is cooled and set, bend both hinges (B for easier removal).

NOTE: Orthosis can be removed and reformed with a heat gun for minor adjustments or reheated in the oven using the re-mold rack.

NOTE: FastForm can get wet during use. If allowed by physician, patient may loosen or remove to dry orthosis using a hair dryer on a cool setting.

STORAGE:

This device is not sterile and should be stored between 4 - 35°C (39-95°F). It must also be protected from strong direct sunlight.

USE AND CARE:

Hand wash in 30°C (86°F) water. Rinse and lay flat to air dry. Note: it is not recommended to use household cleaners or personal hygiene products directly on the orthosis.

MATERIAL CONTENT:

Polymer, Polyester, Polyamide, Elastane

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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