
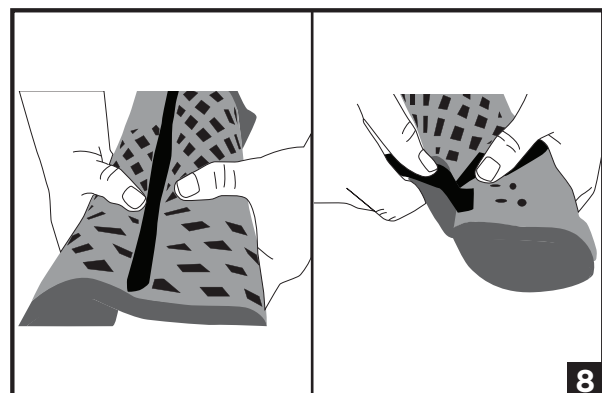
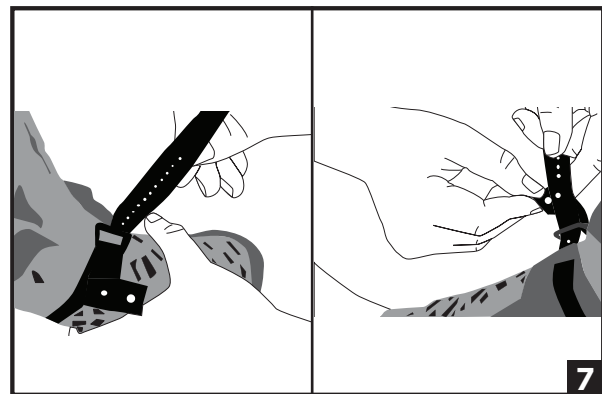
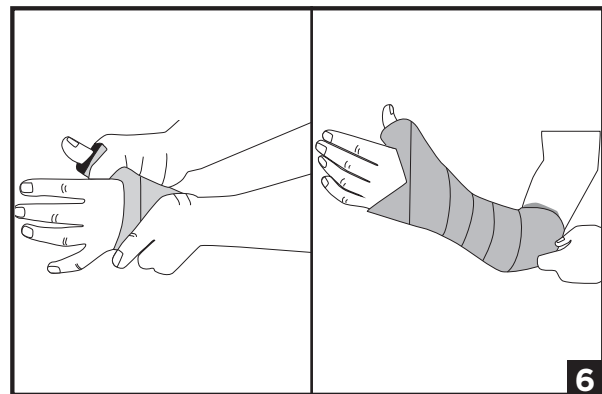
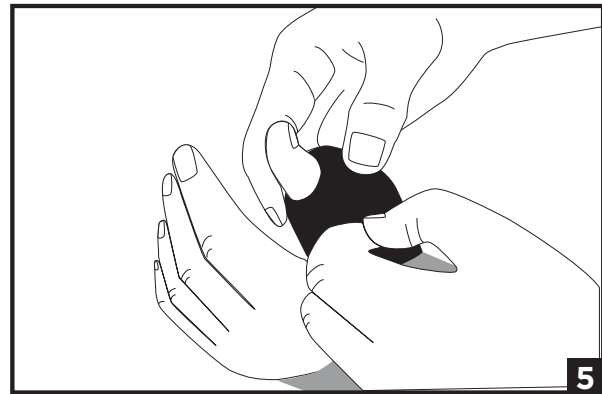
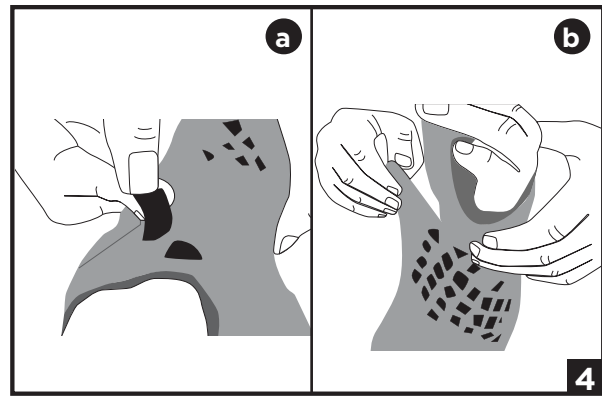
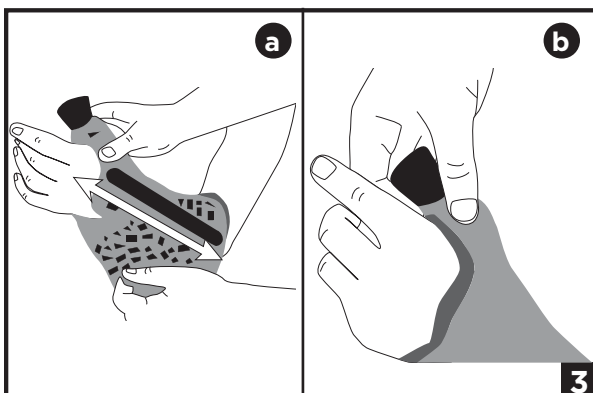
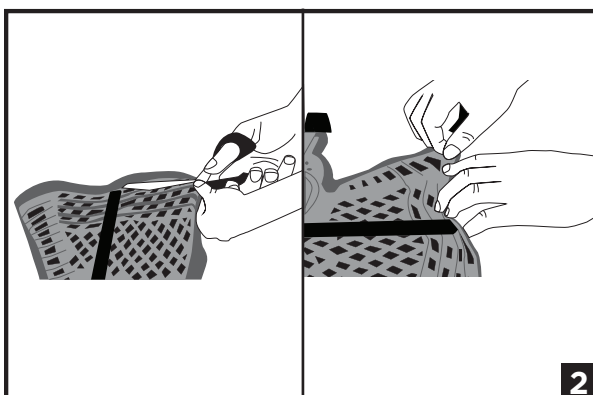
Circumference	M1 - Ulna Bone at Wrist	M2 - Widest point of Forearm
<b>XS</b>	125mm 5.0" 150mm 6.0"	137mm 5.4" 220mm 8.7"
<b>SM/M</b>	145mm 5.7" 190mm 7.5"	150mm 6.0" 260mm 10.0"
<b>M/L</b>	170mm 6.75" 200mm 8"	220mm 8.5" 330mm 13"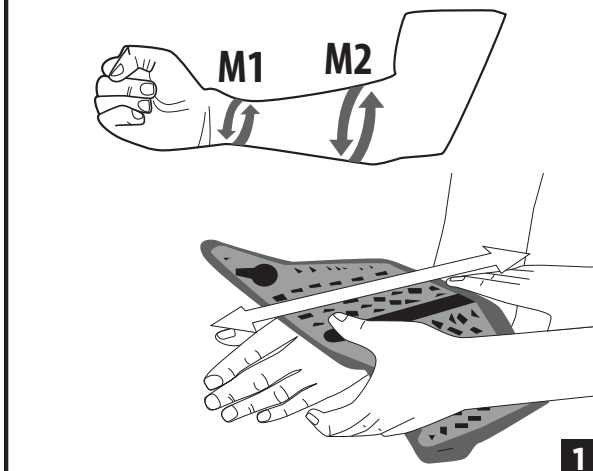

## ENGLISH

### INTENDED USE/INDICATIONS:

Treatment of non-displaced fractures of scaphoid and bones of the MCP joints which may co-occur with fractures of the wrist Ulnar ligament injury, inflammatory conditions such as De Quervain syndrome Provide support in the post-healing rehab phase.

### INTENDED USERS:

The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Complex, compound and/or comminuted fractures.

### ⚠️ WARNINGS AND PRECAUTIONS:

- Device to be fitted by a qualified healthcare professional.
- This Device should not be used on a patient with polyester or elastane allergy.
- Ensure the orthosis is not too hot prior to application.
- Do not apply over open wound.
- Remove the device before any re-molding with a heat-gun, to avoid burning the patient.
- The orthosis will lose its shape above 57°C/135°F or under extreme sunlight. Cover with stockinet or clothing.
- Avoid water or vapor above 40°C/104°F for prolonged periods. Avoid heat sources (e.g. ovens, flames, sunny car windows, very humid conditions) for long periods.
- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your licensed health care professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner. The frequency and duration of use shall be prescribed by a healthcare professional.

### INSTRUCTIONS FOR USE:

1. Measure the circumference of the wrist (M1) and forearm (M2) with a tape measure and select correct size using the measurement chart. **NOTE:** To pre-size the orthosis, place it on the patient's arm, aligning the distal dorsal end parallel to the knuckles and measure from the end of the orthosis to the crease on the inside of the elbow. It should be approximately 3 finger-widths away from the crease. Note how many strips (if any) need to be cut off after heating.

**HEAT:** Place in the FastForm Oven, flat, with the fabric side down and heat until it is soft and malleable. **NOTE:** Heating time may vary (5-10 min) depending on whether you are using a FastForm oven or other Convection oven.

**Primo Oven:** 5 mins at 260°F (127°C)

**Pro Plus/Elite Ovens:** 5 mins at 220°F (105°C)

**Preheated Convection Oven:** 5-10 minutes at 167-185°F (75-85°C)

Do not heat more than one hour.

2. Trim, if required, at the trim sections (C) with scissors while the orthosis is hot. Fold and stack the last level onto the next level and compress to form a comfortable cuff.

3. **a)** Fit the spica over the thumb and align the radial hinge (D) along the radius.  
**b)** Slide the base of the spica well down around thumb and ensure the thumbhole bridge (I) is snugly positioned.

4. **a)** Stretch & wrap the palmar cuff (E) around the hand and align the closure tab (G) onto the closure fixing zone (F) & press into place.  
**b)** Stretch & wrap wing (B) around the arm to overlap wing (A) over its full length. Align & secure yellow dot (N) with yellow dot (M). Ensure semi-circular tab (H) overlaps palmar area (J).

**NOTE:** Easy-fix tabs (I) will temporarily secure wing in place.

5. Roll the fabric liner (L) protruding from the Spica, back over itself to form a padded edge, typically ensuring patient can flex IP joint.

6. Wrap a damp elastic wrap around the wrist and hand snugly but not too tight and mold the orthosis to the correct position. Once the orthosis is cooled and rigid, remove elastic bandage. Verify that the wing (B) and palmar cuff (E) are free to allow for subsequent adjustment.

7. Apply the Compliance strap to the wrist area, by sliding the strap through the slot in the liner and tighten it. Attach the forearm strap. To lock the compliance strap, push the white fastener to matching hole.

8. If the brace is removed for rehab, flex radial (D) and palmer hinge for easier removal.

**NOTE:** FastForm can get wet during use. If allowed by physician, patient may loosen or remove to dry orthosis using a hair dryer on a cool setting.

### STORAGE:

This device is not sterile and should be stored between 4 - 35°C (39-95°F). It must also be protected from strong direct sunlight.

### USE AND CARE:

Hand wash in 30°C (86°F) water. Rinse and lay flat to air dry. Do not machine wash or tumble dry. **NOTE:** it is not recommended to use household cleaners or personal hygiene products directly on the orthosis.

### MATERIAL CONTENT:

Polymer, Polyester, Polyamide, Elastane

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

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FastForm is a trademark of Breg, Inc.
© 2025 Breg, Inc. All rights reserved.