
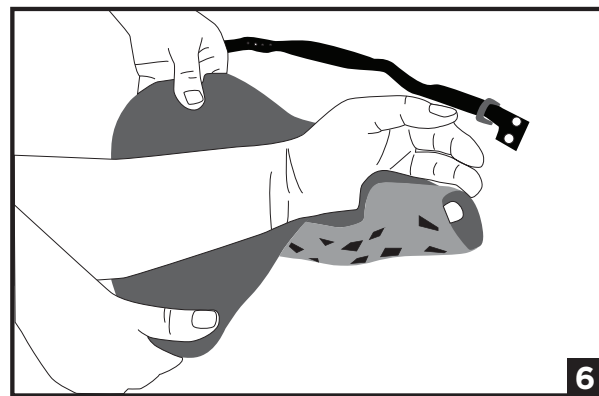
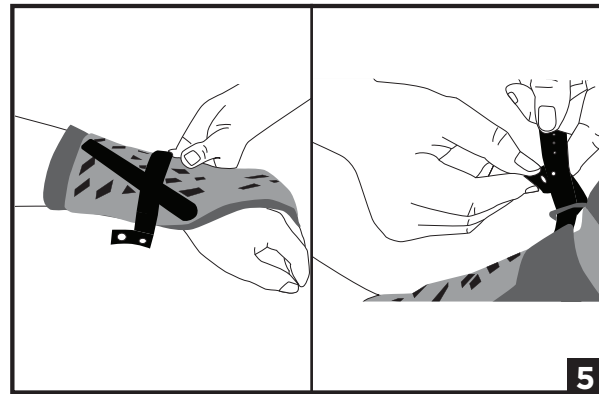
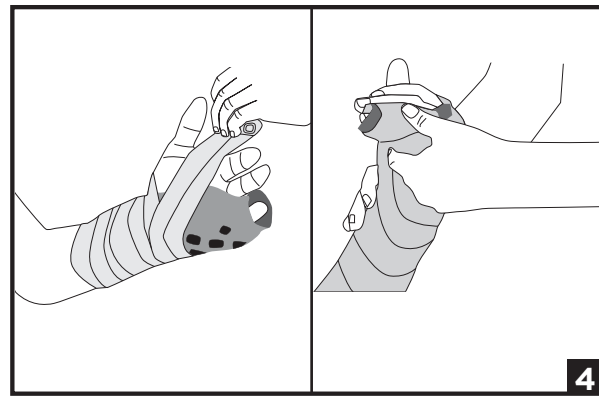
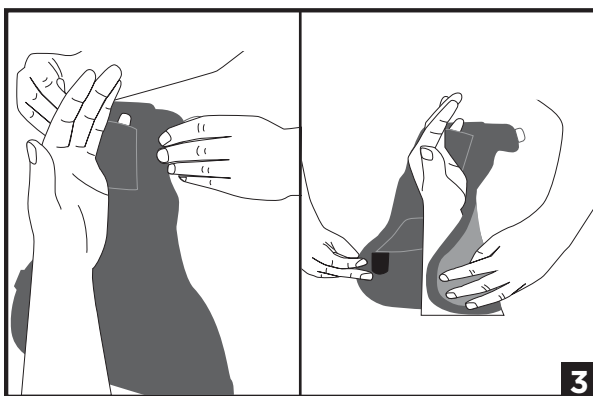
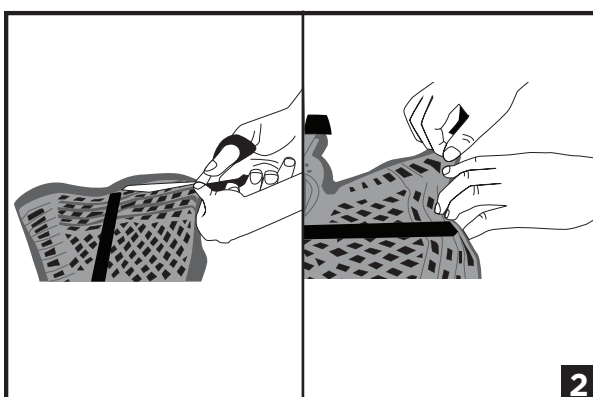
Circumference	M1 - Ulna Bone at Wrist	M2 - Widest point of Forearm
<b>XS</b>	125mm 5.0" 150mm 6.0"	137mm 5.4" 220mm 8.7"
<b>SM/M</b>	145mm 5.7" 190mm 7.5"	150mm 6.0" 260mm 10.0"
<b>M/L</b>	170mm 6.75" 200mm 8"	220mm 8.5" 330mm 13"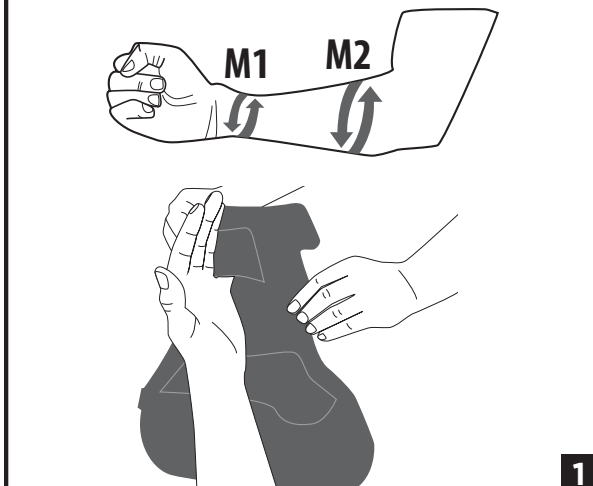

## ENGLISH

### INTENDED USE/INDICATIONS:

Treatment of minimally-displaced fractures of the metacarpals, sprains and strains, tendonitis and other cumulative trauma injuries. Provide support in the post-healing rehab phase.

### INTENDED USERS:

The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Complex, compound and/or comminuted fractures.

### ⚠️ WARNINGS AND PRECAUTIONS:

- This Device should not be used on a patient with polyester or elastane allergy.
- Ensure the orthosis is not too hot prior to application.
- Do not apply over open wound.
- Remove the device before any re-molding with a heat-gun, to avoid burning the patient.
- The orthosis will lose its shape above 57°C/135°F or under extreme sunlight. Cover with stockinet or clothing.
- Avoid water or vapor above 40°C/104°F for prolonged periods. Avoid heat sources (e.g. ovens, flames, sunny car windows, very humid conditions) for long periods.
- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner. The frequency and duration of use shall be prescribed by a healthcare professional.

### INSTRUCTIONS FOR USE:

1. Measure the wrist with a tape measure and select the correct size from the sizing chart. **NOTE:** Select radial or ulnar treatment configuration. Radial configuration is achieved by pushing out the thumbhole plug (F) and pushing fabric liner through the pre-cut thumbhole slot. To pre-size align the distal edge of the orthosis with the desired end-point on the finger and not how much to trim if necessary. **HEAT:** Place in the FastForm Oven, flat, with the fabric side down and heat until it is soft and malleable. **NOTE:** Heating time may vary (5-10 min) depending on whether you are using a FastForm oven or other Convection oven. **Primo Oven:** 5 mins at 260°F (127°C) **Pro Plus/Elite Ovens:** 5 mins at 220°F (105°C) **Preheated Convection Oven:** 5-10 minutes at 167-185°F (75-85°C) Do not heat more than one hour.
2. Trim, if required, at the trim sections (C) with scissors while the orthosis is hot. Fold and stack the last level onto the next level and compress to form a comfortable cuff.
3. Place required fingers into the finger sock (D), close the wrist section using the easy-fix tab (A). Wrap the fingers using the easy-fix tab (B), making sure the finger support closure is between the injured and uninjured fingers. Align the fingers in desired position.
4. Wrap a damp elastic bandage firmly but not too tight proximal to the distal orthosis. Mold to the desired position.
5. Once the orthosis is cooled and rigid, remove elastic bandage. Apply the Compliance strap to the wrist area, by sliding the strap through the slot in the liner and tighten it. Attach the forearm strap. To lock the compliance strap, push the white fastener to matching hole.
6. To prepare orthosis for rehab use, remove it & flex both angled hinges G fully for easy opening. **NOTE:** FastForm can get wet during use. If allowed by physician, patient may loosen or remove to dry orthosis using a hair dryer on a cool setting.

### STORAGE:

This device is not sterile and should be stored between 4 - 35°C (39-95°F). It must also be protected from strong direct sunlight.

### USE AND CARE:

Hand wash in 30°C (86°F) water. Rinse and lay flat to air dry. Do not machine wash or tumble dry. Note: it is not recommended to use household cleaners or personal hygiene products directly on the orthosis.

### MATERIAL CONTENT:

Polymer, Polyester, Polyamide, Elastane

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

### DISPOSAL:

Dispose of product according to local regulations.

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© 2025 Breg, Inc. All rights reserved.
FastForm is a trademark of Breg, Inc.