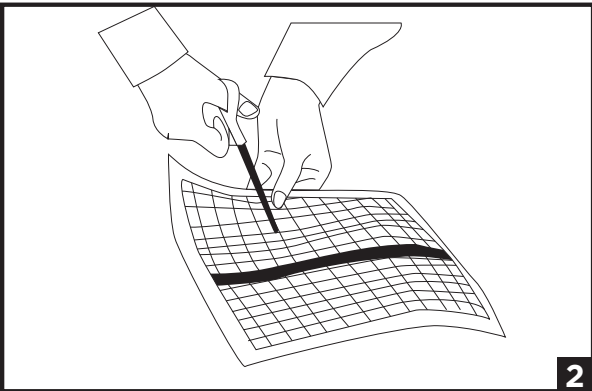
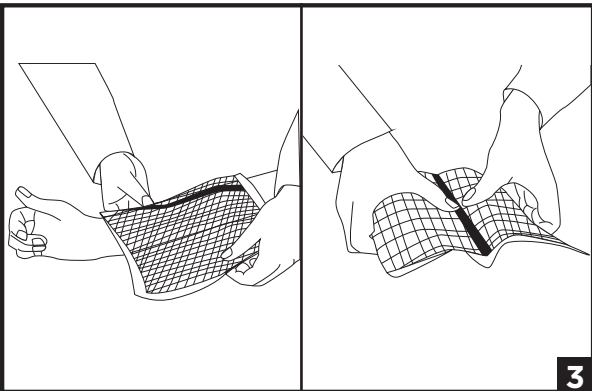
1

2

3

ENGLISH

INTENDED USE/INDICATIONS:

The sheet is indicated for use by a trained healthcare professional in the custom crafting of bespoke orthopedic supports including extensions to other FastForm® Products; hand, wrist, finger and thumb splints; orthopedic orthosis, external immobilization devices and rehabilitation aids; serial splints; outrigger bases; spasticity splints; functional position splints, resting mitt splints.

INTENDED USERS:

The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Complex, compound and/or comminuted fractures and load bearing applications.

⚠️ WARNINGS AND PRECAUTIONS:

- The VERSI Sheet is designed for use by a qualified healthcare professional.
- Patients should consult a medical professional immediately if they experience any pain, swelling, sensation changes, or unusual reactions while using their device.
- This device should not be used on a patient with polyester or elastane allergy.
- Ensure the crafted device is not too hot prior to application.
- Do not apply over open wound.
- Remove the device before re-molding with heat-gun, to avoid burning patient.
- The crafted device will lose its shape above 57°C/135°F or under extreme sunlight. Cover with stockinet or clothing.
- Avoid water or vapor above 40°C/104°F for prolonged periods. Avoid heat sources (e.g. ovens, flames, sunny car windows, very humid conditions) for long periods.
- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- Federal law restricts this device to sale by or on the order of a licensed health care practitioner. The frequency and duration of use shall be prescribed by a healthcare professional.
- For single patient use only.

INSTRUCTIONS FOR USE:

- Place the VERSI™ Sheet (fabric side down) in the FastForm Oven. Ensure the plastic does not touch the sides. Heat 5-10 minutes depending on the type of oven(FastForm or Convection) Check that all areas are fully malleable. If not, reheat in 1-minute increments. **NOTE:** If more than one sheet is needed. Remove backing paper from sheet. These areas can be used for tessellation of multiple sheets.
Primo Oven: 5 mins at 260°F (127°C)
Pro Plus/Elite Ovens: 5 mins at 220°F (105°C)
Preheated Convection Oven: 5-10 minutes at 167-185°F (75-85°C)
 Do not heat more than one hour.
- Trim the sheet to the desired shape, with scissors while the sheet is hot. Fold, stack and compress to form a comfortable cuff.
- Ensure that the fabric side of the sheet is applied to the patient's skin. The plastic side should never touch the skin.

NOTE: FastForm can get wet during use. If allowed by physician, patient may loosen or remove to dry orthosis using a hair dryer on a cool setting.

STORAGE:

This device is not sterile and should be stored between 4 - 35°C (39-95°F). It must also be protected from strong direct sunlight.

USE AND CARE:

Hand wash in 30°C (86°F) water. Rinse and air dry. Do not machine wash or tumble dry. Note: it is not recommended to use household cleaners or personal hygiene products directly on the crafted device.

MATERIAL CONTENT:

Polymer, Polyester, Polyamide, Elastane

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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FastForm and VERSI are trademarks of Breg, Inc.
© 2025 Breg, Inc. All rights reserved.