## ENGLISH

### INTENDED USE/INDICATIONS:

Foot and Ankle products provide overall support, compression and/or protection to the lower leg including the ankle, and foot after an injury or surgery.

Indicated to increase support in the inversion and eversion control on chronically unstable ankles, and for increased inversion and eversion control for grade 1 and 2 acutely sprained ankles where dorsiflexion and plantar-flexion is permissible and inflammation is under control.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Contraindicated for immediate application to acute grade 3 ankle sprains where plantar-flexion and dorsiflexion is not desirable or acceptable. Application is appropriate after inflammation has been controlled and plantar-flexion and dorsiflexion is again possible starting in the early weight bearing phase of rehabilitation.

### ⚠ WARNINGS AND PRECAUTIONS:

- For Single Patient Use Only.
- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- This device will not prevent or reduce all injuries.
- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.
- This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150°F (65° C.) for any extended period of time. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

### INSTRUCTIONS FOR USE:

#### 1. Detach Over-Center Buckles

Open the over-center buckles by pinching the two gray tabs on the buckles.

#### 2. Place in Shoe

- Place brace in the shoe. It can be placed on top of the shoe innersole or under a thin innersole. Push footplate of the brace into the heel of the shoe.
- Tilt the uprights back toward the heel.

#### 3. Place Foot into the Shoe

With the shoe fully unlaced, slip foot into the shoe.

#### 4. Detach the Uprights

Detach the hook and loop fastener at the back of the calf where the two uprights are held together.

#### 5. Position the Uprights

- Align the medial upright with the ankle bone.
- While holding the medial upright, align the lateral upright and attach the hook and loop fastener at the back of the calf.

#### 6. Fasten Upper Strap

- Wrap the upper strap around the leg and connect the over-center buckle.
- Adjust the strap tension using the hook to pile fastener at the strap end.

#### 7. Fasten Lower Strap

- Connect the over-center buckle on the opposite upright.
- Adjust the strap tension using the hook to pile fastener at the strap end.

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

### MATERIAL CONTENT:

Ethylene Vinyl Acetate (EVA), Nylon, Polyester, Stainless Steel, Steel

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

### NOTICE TO USERS:

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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