## ENGLISH

### INTENDED USE/INDICATIONS:

Foot and ankle products provide overall support, compression and/or protection to the lower leg including the ankle and foot after an injury or surgery. Indicated for chronic ankle instability, sprains or strains, and protection against inversion and eversion ankle injuries.

### INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Contraindicated for fractures of the foot and ankle.

### ⚠ WARNINGS AND PRECAUTIONS:

- This device is offered for sale by or on the order of a physician or other qualified medical professional.
- Do not leave this device in the trunk of a car on a hot day or in any other period where the temperature may exceed 150°F (65°C) for any extended period of time. If any additional pain or symptoms occur while using this brace, discontinue use and seek medical attention.
- For Single Patient Use.

### INSTRUCTIONS FOR USE:

- Position the brace. Slip foot into brace ensuring that the tongue is between the top of the foot and the laces. Pull tabs and hook to secure.
- Pull the uprights apart so the foot can be positioned between the uprights.
- Place the foot on the footplate so the ankle joint is aligned with the hinges. The pivot point needs to align with the ankle (or malleolus) bone.
- Press the paddle against the leg so the hook and loop patch attaches to the brace. This will lock the lateral upright into position. Repeat this process on the medial side.
- Once hinge is secured, slightly invert foot, wrap the medial strap (inside strap) across the top and then under the foot back to the medial side. Pull the strap vertically until firm medial (inner) tension is achieved. Secure strap to the medial side.
- Evert foot, wrap the lateral strap (outside strap) across the top and then under the foot back to the lateral side. Pull the strap vertically until firm lateral (outer) tension is achieved. Secure strap to the lateral side.
- Wrap and secure the Calf Strap. Wrap upper strap around the lower calf (top of the brace) and secure.

### USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

### MATERIAL CONTENT:

Brass, Nylon, Polyester, Polypropylene, Polyvinyl Chloride (PVC), Steel

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

### DISPOSAL:

Dispose of product according to local regulations.

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