





SlingShot® 2 *Shoulder Brace*

Patient Application Instruction Sheet

1.

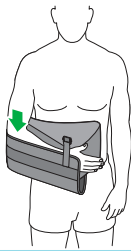


2.

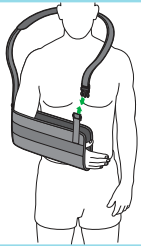


3.




4.

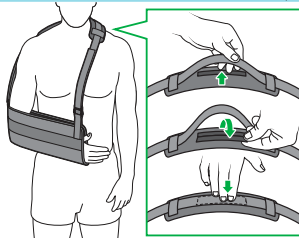


5.

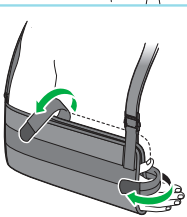


6.

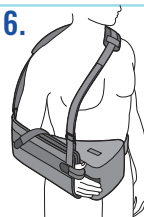


7.

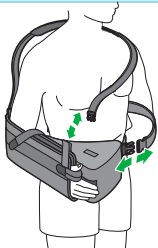


8.

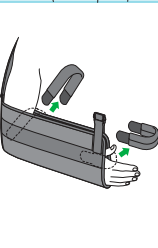

8.

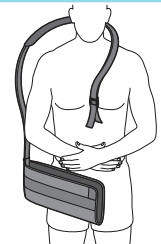

9.

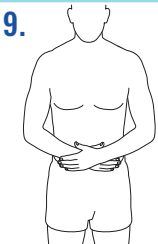

Donning (Reapplying Brace)

1. While supporting the injured arm, slide the forearm into the sling, ensuring your elbow fits snugly into the corner of the sling pocket. Make sure the abduction pillow is properly placed at waist level per your physicians recommendations.
2. Wrap the shoulder strap around the neck and shoulder and attach strap to the clip/buckle at front of sling.
3. Wrap the waist strap around waist and secure with the clip buckle.
4. Adjust the straps and shoulder pad as needed to ensure a comfortable fit that immobilizes your shoulder without causing pain or restricting circulation.
5. Attach the forearm closure strap across the top of the sling. Attach the thumb strap to open end of the sling between forefinger and thumb. Both straps help maintain position of forearm in sling.
6. Double-check the brace is aligned properly and your arm is supported as prescribed.

Doffing (Removing Brace)

7. While seated, unclip the waist and shoulder strap first. Then, undo the thumb and forearm closure straps on sling.
8. Support your injured arm with your other hand, then gently slide your arm out of the sling.
9. Remove the brace completely, avoiding sudden movements or strain to the shoulder.
10. Store the Slingshot 2 in a clean, dry place until your next use.

Additional Tips

- Always follow your healthcare provider's instructions regarding brace wear time and activity restrictions.
- If you experience increased pain, numbness, tingling, or swelling, remove the brace and contact your provider immediately.
- Clean the brace according to the manufacturer's guidelines to maintain hygiene and function.

Washing Instructions

Straps and sling should be removed and cleaned regularly depending on your activity level and perspiration. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

⚠ Warnings and Precautions

- For single patient use only.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program.
- If you experience increased pain, swelling, or any adverse reactions while using this product, immediately consult your medical professional.
- Carefully read fitting instructions and warnings prior to use.

Learn More



Breg.com/ss2