## ENGLISH

### INTENDED USE/INDICATIONS:

Intended to provide relief from unicompartamental knee osteoarthritis. Indicated for degenerative meniscal tears, acute meniscal tears, meniscal repairs, and other knee conditions that may benefit from unicompartamental unloading.

### INTENDED USERS:

The intended user should be a licensed professional, the patient or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### CONTRAINDICATIONS:

Not Applicable.

### WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions. Failure to properly position the brace and fasten each strap will compromise performance and comfort.
- For single patient use only.
- If you experience increased pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

- Care and cleaning of this product is essential to its continuing strength and performance. Follow removal and cleaning process described in instructions.
- When using braces bilaterally, modify activity as necessary to keep medial hinges from contacting each other.
- When using the Freestyle Plus OA brace it is important to keep dirt and grit out of the hinge mechanism to maintain proper hinge mobility and brace function.

### INSTRUCTIONS FOR USE:

**IMPORTANT:** Make sure the condyle pad is properly aligned with the hinge thumbwheel dial before applying the brace to the knee.

- Prior to application, set the hinge to 0.
- Sit with the knee bent at 30°.
- Position the brace by placing the hinge on the unaffected side of the knee. The hinge should be aligned with the center of the knee.
- Maintain the brace position with one hand while using the other hand to pull tab #1 across the upper shin and fasten securely. Finger pockets can be used on strap tabs 1-4 to help fasten tabs.
- Pull tab #2 across the lower thigh and fasten securely.
- Pull tab #3 across the lower shin and fasten securely.
- Pull tab #4 across the upper thigh and fasten securely. Make sure all tabs are fastened securely and have full contact with fabric below. To avoid rotation of the brace during application, secure the brace with one hand while tensioning the last two straps.
- Pass the lower strap (#5) through the plastic ring on the calf, lay the strap onto itself, and fasten securely.
- Repeat for the upper strap (#6).
- Once the brace is properly applied, rotate the thumbwheel to adjust brace off-load.
- HINGE ADJUSTMENT:** Turn the thumbwheel dial clockwise to increase off-load and counterclockwise to reduce the off-load. The hinge settings window displays settings ranging from 0 to 10. Always follow your medical professional's recommendation regarding the proper hinge setting.
- CONDYLE PAD ADJUSTMENT:** For a more snug fit at the knee, add shims to the condyle pad.
  - Bend the brace at the hinge to access the condyle pad.
  - Remove the condyle pad from the hinge.
  - Attach the desired number of shim(s) to the condyle pad.
  - Re-attach the condyle pad to the brace.

### USE AND CARE:

Make sure brace is free of any debris (hair, dirt, etc) prior to wearing. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Apply a dry lubricant such as a Teflon spray to the hinge after washing. DO NOT machine wash or tumble dry.

### MATERIAL CONTENT:

Aluminum, BreatheFit™, Nylon, Polyester, Stainless Steel, Silicone.

### WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see [www.breg.com](http://www.breg.com)

### DISPOSAL:

Dispose of product according to local regulations.

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